**Bedienungsanleitung**  
**Working instructions**  
**Mode d'emploi**

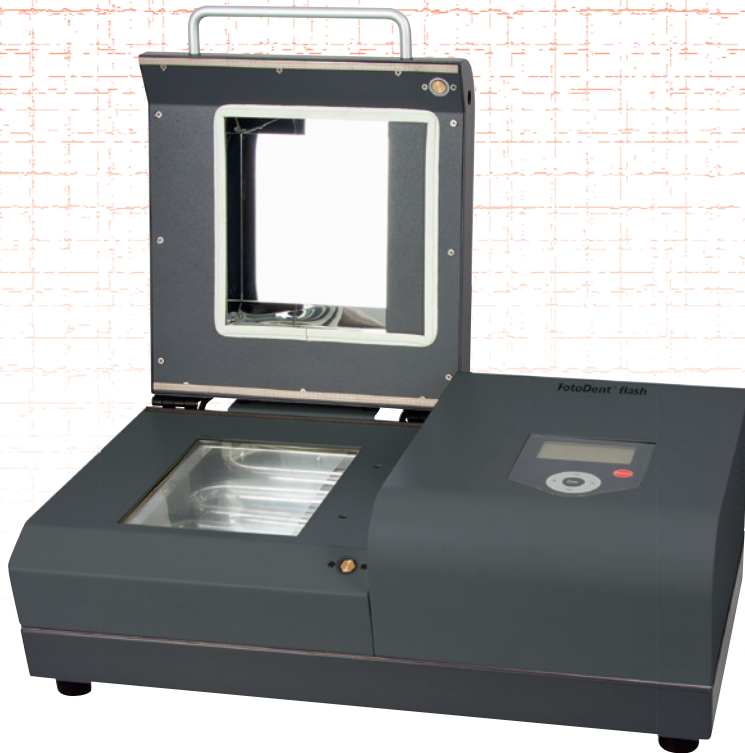
deutsch

english

français

# FotoDent® flash

**Lichtpolymerisationsgerät**  
**Light polymerisation unit**  
**Appareil de photopolymérisation**

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## Placing the product out of operation

This unit is subject to regulations of the WEEE (waste electrical and electronic equipment), the return of the unit is granted by public waste management organisations. This regulation is valid for all member states of the European Union.

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This unit is subject to regulations of the WEEE (waste electrical and electronic equipment), the return of the unit is granted by public waste management organisations. This regulation is valid for all member states of the European Union.

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**Contents** (EN)

|  |       |
|--|-------|
| Placing the product out of operation   | 2     |
| Exploded drawing                       | 18    |
| Spare parts list                       | 19    |
| Technical data                         | 20    |
| Flash tube spectrum                    | 20    |
| Safety instructions                    | 21    |
| Description of the unit                | 22    |
| Putting into service                   | 22    |
| Information on using<br>protective gas | 22    |
| Control lamps                          | 23    |
| Operation                              | 24    |
| Exchange of flash tubes                | 27    |
| Care and maintenance                   | 27    |
| General notes                          | 27    |
| Guarantee                              | 28    |
| Trouble shooting                       | 29–30 |
| Declaration of conformity              | 47    |
| Wiring diagram                         | 49–51 |

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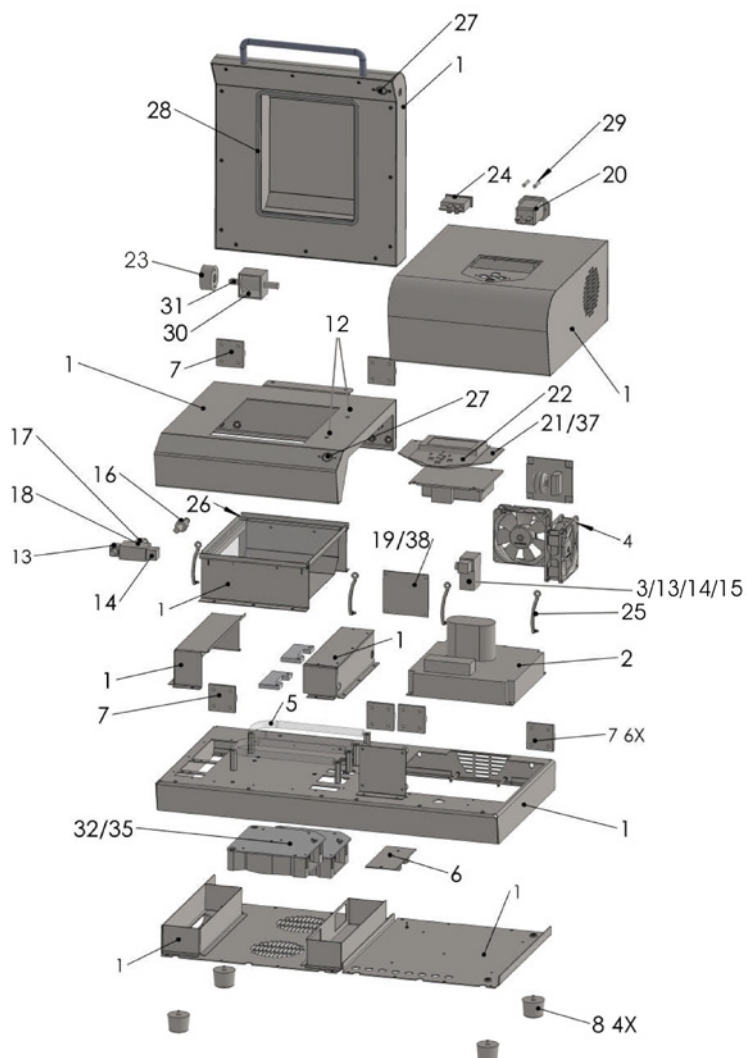
## Exploded drawing

## Spare parts list

|                             | <b>Item No.</b>                             |                   |
|-----------------------------|---|-------------------|
| 1                           | Housing complete with reflector             | 5900-1            |
| 2                           | Power supply                                | 5700-10WS         |
| 3                           | Y-screw connect                             | 5719              |
| 4                           | Fan, 12 V                                   | 5706              |
| 5                           | Flash tubes 2 pcs. with fixing element      | 5901-3KPL         |
| 6                           | Ignition transmission plate with spark coil | 5708kpl           |
| 7                           | Hinge                                       | 50600             |
| 8                           | Housing bottom                              | 51166             |
| 12                          | Angle swivelling screw-fitting              | 50267MS           |
| 13                          | Connection                                  | 51050             |
| 14                          | L-angle                                     | 50288             |
| 15                          | Magnetic valve                              | 5526R             |
|                             |   | 5525-265WR        |
| 16                          | Coupling with external screw thread         | 50097             |
| 17                          | Plug nipple with external screw thread      | 50095             |
| 18                          | Needle valve                                | 50937             |
| 19/38                       | Relay circuit board                         | ES#2364.089.00    |
| 20                          | Line filter                                 | 5713              |
| 21/37                       | Electronic timer (230/115V)                 | 5760FLASH         |
| 22                          | Adhesive foil for electronic timer          | 5582EVO           |
| 23                          | Mount lift magnet                           | 57019             |
| 24                          | Rocker switch, red                          | 50847             |
| 25                          | Ground strap / lid                          | 5743              |
| 26                          | Glass plate                                 | 59010             |
| 27                          | Magnet and Reed magnetic switch             | 53014/53014MAGNET |
| 28                          | Cover rubber                                | 51206             |
| 29                          | Micro fuse                                  | 50908             |
| 30                          | Stroke magnet                               | 5702              |
| 31                          | Lid interlock                               | 57022             |
| 32                          | Radial ventilator (230 V)                   | 50899             |
| <b>Only for 115 V/60 Hz</b> |   |                   |
| 35                          | Radial ventilator (115 V)                   | 50950             |
| 21/37                       | Elektronic timer (230/115V)                 | 5760FLASH         |
| 19/38                       | Relay circuit board                         | ES#2364.089.00    |
| 39/41                       | Connecting cable (115 V/60 Hz)              | D-6559091         |
| 40                          | Unit plug                                   | 51163             |
| <b>Only for 220 V/60 Hz</b> |   |                   |
| 41                          | Connecting cable (220 V/60 Hz)              | D-6559091         |
| 42                          | Unit plug                                   | D-51163           |
| <b>Only for 230 V/50 Hz</b> |   |                   |
| 43                          | Connecting cable (230 V/50 Hz)              | D-6559090         |

# FotoDent® flash

## Technical Data

H x W x D 195 x 517 x 295 mm  
polymerisation chamber  
40 x 140 x 180 mm

Weight 16.6 kg

### FotoDent® flash

Voltage 230 V / 50 Hz, 220 V / 60 Hz, 115 V / 60 Hz  
(voltage/frequency refer to type plate)

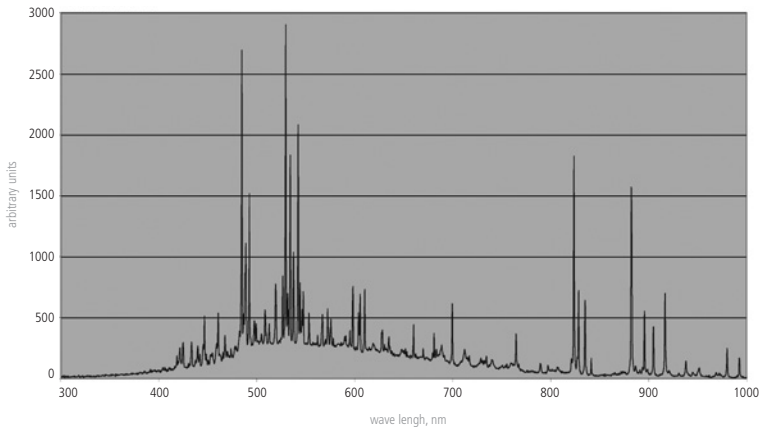
Power input 550 W

Maximum operating pressure 5 bar (70 psi)

Flash frequency 10 Hz  $\pm$  2 Hz

## Flash tube spectrum

FotoDent® flash

## Safety instructions

ⓘ **Attention!** Read the following notes before connecting and putting the unit into service. The operating service and the function of the unit can only be guaranteed for, if the general safety and accident prevention instructions of the legislation as well as the safety instructions of the working instructions are observed.

1. This unit must only be used according to the present working instructions. We cannot be held liable for damages which have been caused by improper treatment resp. faulty manners of operation.
2. The indicated voltage on the type plate has to meet the voltage of the power source.
3. The unit must only be connected with a socket with protective conductor. Never touch the plug with wet hands.
4. Put the unit on a stable and even basis.
5. Do not intrude any objects into the unit outside of the polymerisation chamber.
6. Only those appliances and spare parts may be used which are released by the manufacturer. We are not responsible for damages caused by the use of external parts.
7. Do not lock the operating and safety elements of the unit.
8. The unit has to be examined before operation in view of proper condition. If the unit is not in proper condition, it must not be used and has to be marked accordingly.
9. Before cleaning and maintenance of the unit or exchange of parts the plug has to be disconnected.
10. Plates and labels have to be maintained in a well legible condition and must not be removed.
11. The opening of the unit and repairs are only to be carried out by accordingly trained experts.
12. Do only use nitrogen or carbon dioxide as protective gas. In no case use flammable or toxic gases. Observe the general safety and accident prevention instructions for the use with the chosen gas.
13. Do only use permitted gas pressure tanks.
14. When connecting the gas pressure tank to the unit check the tightness of the supply pipe. Make sure that the gas pressure tank cannot tip over.
15. Never connect the gas tank without using the pressure reducer, the unit has to be operated with a maximum pressure of 5 bar (70 psi).
16. If the unit is not used, block the gas supply, do not operate the unit unattended.
17. Provide sufficient ventilation in the working area.
18. User of unit only with delivered connecting cable and unit plug.

# FotoDent® flash

## Description of the unit

FotoDent® flash is a professional light polymerisation unit for post-curing of FotoDent®- materials by using protective gas. The protective gas atmosphere prevents from the development of an inhi-

bitation layer on the surface of FotoDent® profiles. The programmable controller of the FotoDent® flash and the intuitive menu navigation enable an easy handling of the unit.

## Putting into service

Before putting into service make sure that no transport damages occurred. Check for external damages at the housing and control the glass plate on the bottom of the polymerisation chamber and the light tubes underneath. Further ensure, that the indicated voltage on the type plate meets the voltage of the power source. Put the with delivered connecting cable and

unit plug into the socket of the unit to set up a power connection. Safely fix the protective gas supply pipe at the hose nozzle of the pressure reducer of the gas pressure tank with the help of the hose clamp. Create a connection to the unit with the quick coupler. The pressure must not exceed 5 bar (70 psi).

## Information on using protective gas

By using protective gas it is possible to do the polymerisation in an atmosphere without oxygen. The protective gas replaces the air in the polymerisation chamber and thus reduces the oxygen content in the polymerisation chamber of the FotoDent® flash leading to a curing of the FotoDent® profiles without inhibition layer.

If the breaker is on „gas“ the protective gas supply is done automatically when starting the polymerisation process. With breaker position „0“ the gas supply is closed.

The flow rate is pre-set (120 l/h), if necessary the adjustment can be changed

at the needle valve at the back of the unit (observe safety instructions). If modifying the output pressure of the bottle at the pressure reducer of the gas pressure tank the flow rate can be changed without changing the needle valve.

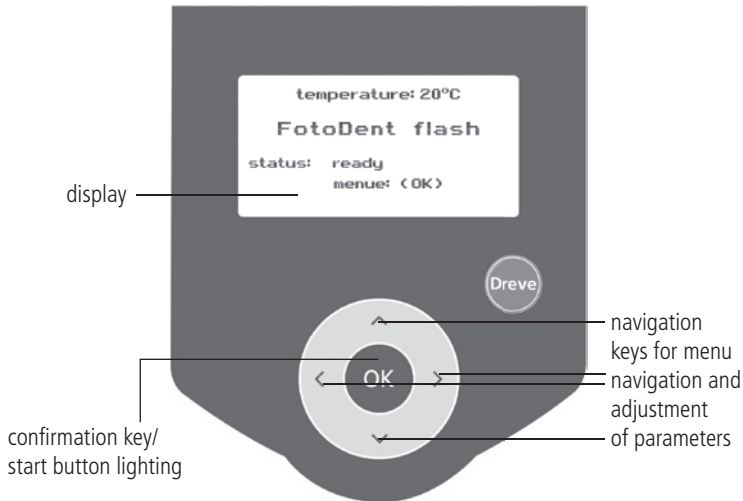
| Pressure |     | Flow rate |
|----------|-----|-----------|
| bar      | psi | l/h       |
| 1        | 14  | 60        |
| 2        | 28  | 80        |
| 3        | 42  | 100       |
| 4        | 52  | 120       |
| 5        | 70  | 140       |

When using nitrogen we recommend to adjust the pressure to 1 bar, when using carbon dioxide 4 bar. The operator is responsible for the warranty of the operating safety of the FotoDent® flash. Observe national safety instructions when using gases for the technical purpose. Personnel working with the FotoDent® flash

has to be instructed about the danger of protective gases. The used protective gas reduces the oxygen content in the working area.

If the FotoDent® flash is adjusted correctly there is no danger for the personnel.

## Controls

# FotoDent® flash

## Operation of the FotoDent® flash

Check the power supply and connect the gas. Then turn on the main switch at the right of the back of the unit. The display shows the following announcement:

```
temperature: 20°C  
FotoDent flash  
status: ready  
menue: <OK>
```

By pushing „OK“ the selection-menu appears:

```
program call  
program provide  
service
```

The menu items are further described below.

### INFO

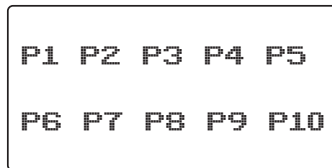
For equal results during postcuring switch on the unit at the beginning of the day respectively working process. Ideally start a lightning process of ca. 5 minutes to bring the flash bulbs on processing temperature.

Between the first and the second lighting process there is a break of lighting. This break is for the cooling down of the parts. The gas valve is opened during the break to support the cooling. After the break the gas valve closes automatically and therefore is not flowing during the second lighting process.


## Selecting a program

Select the menu item „select program“ by using the arrow keys. Confirm with “OK”. The display now shows the storage spaces, provided that there are programs stored yet:

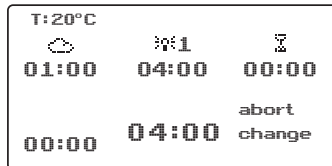Select the needed program with the arrow keys and confirm with “OK”. The parameter of the program are shown in the display:

The total time display of the chosen program is blinking. Confirm the start of the program with „OK“. The Display shows “lid open”:

Open the lid and put the object onto the glass plate of the polymerisation chamber. Close the unit. The lighting process starts automatically according to the given parameters. The total time is counted down during the lighting process, “Abort” (Cancel) is blinking. After the program has expired the display shows that the drawer is open. Get back to the selection menu by pressing “OK ” for some seconds.

### INFO

If the program has been started by accident, it can be terminated ahead of time by pushing “Abort” (cancel).

In the running program the function “Abort” (cancel) is pre-selected and blinking.

Cancel the program by pressing “OK” very long (ca. 2 seconds). Again, the selection display is shown.

Changes of the parameters of the program cannot be made while the program is running.

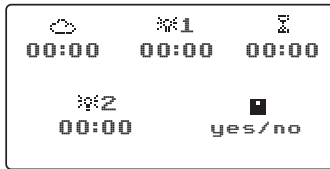
### INFO

If the program has to be quit without starting it, the “OK” key has to be pressed for about 2 seconds. Afterwards the starting menu is shown again.

# FotoDent® flash

## Generating a program/editing a program

Chose the menu item "Generating a program" in the main menu by using the arrow keys and confirm the choice with „OK“. The following display is now shown:

- ☁ Time for pre-gassing
- ☼1 Time for the first light exposure process
- ⌚ Interval between the first and second light exposure process
- ☼2 Time for the second light exposure process
- Save programme

Select a particular parameter of the program with "OK". According to the current parameter of the program you have to give in the desired duration by using the arrow keys. If one parameter should be dropped, set the duration on 00 minutes, 00 seconds. After running through all necessary parameters the symbol disk is blinking to save the parameters. Press "OK", „No" is blinking. Choose the desired option by using the arrow keys, confirm with ok.

### INFO

The light exposure times mentioned in the list represent a recommenda-

tion for the respective Dreve Dentamid products.

| Product           | Light exposure (1) |
|-------------------|--------------------|
| FotoDent® guide   | 10 min             |
| FotoDent® cast    | 10 min             |
| FotoDent® model   | 12 min             |
| FotoDent® tray    | 12 min             |
| FotoDent® gingiva | 12 min             |
| FotoDent® denture | 12 min             |

If you chose "Yes", you have to choose the storage space in the next step. There are 10 spaces available for saving, the corresponding number of the storage space appears, as soon as the program is stored in this space. Later on, the program can be recalled any time by this number. Confirm the selection of the storage space with "OK". Afterwards, the menu for selecting the program appears.

Select the last saved program by selecting the before chosen storage space. For the changing of a program select in the chosen program "change" by using the arrow keys and confirm with "OK". Now shown is the menu for the input of the parameters of the program. The saving of changes happens according to the storage of a new program.

After selecting the service menu the display shows the following:

The screenshot shows a menu with the following items:

- P1 P2 P3 P4 P5
- P6 P7 P8 P9 P10

With the service menu the following options can be adjusted or recalled:

```
language.....: (en)
cooling temperature: 30°C
operating time.....: 0000h
gas valve.....: OFF
temp. sensor 1/2 : OK / OK
uv-bulk 1/2.....: OK / OK
```

- Adjustment of language (selection german/english)
- Adjustment of the cooling temperature (individual adjustable)
- Display and resetting of operating hours
- Function check of the gas valve
- Function display of both temperature sensors
- Function display of the flash bulbs

## Adjustment of language

For changing the language choose by using the arrow keys the option, confirm with "OK". By using the arrow keys

select between "de" for German and "en" for English, confirm your selection with „OK“.

## Adjustment of the cooling temperature

The temperature, at which the unit has to start the cooling process, is individual adjustable. The adjustment range is set between 30 °C and 60 °C and can be customized individually to the ambient temperature. For adjusting this temperature choose the option by using the arrow keys and confirm with „OK“. To enter the data of the temperature put in the figures by using the arrow keys up and down. To switch between the positions of the figures use the arrow keys left and right.

Display and resetting of the operating hours

The operating hours of the flash bulbs can be displayed with this menu item. The hours are displayed automatically. After a change of the flash bulbs it is recommended to reset the operating hours for a better control of the life cycle of the flash bulbs. Therefore choose the menu item "operating hours" and confirm with "OK". To reset the operating hours press the left and right arrow key simultaneous until the operating hours are blinking. Now press "OK", the operating hours will be resetted automatically.

Checking the function of the gas valve

# FotoDent® flash

To ensure that the gas valve functions well, the function can be checked with this menu item. Therefor press "OK". The gas valve opens shortly with a "clickclack" sound and then closes itself again. The gas valve is defect when there is no sound.

The function of the temperature sensors and the flash bulbs can also be seen in the service menu. The parts are all right when the status is shown as "OK".

## Exchange of flash tubes

**Before exchanging the flash tubes pull the plug and wait at least 10 minutes!**

Put the unit vertically onto the side and unscrew the ground lid at the side of the light chamber. Inside of the unit on the right, there are three allen screw, which have to be unscrewed with the allen screwdriver provided. Afterwards put the unit again in the horizontal position and open the lid and interlid. Take out the glass plate! On the right of the glass plate, unscrew the cover

with 5 recessedhead screws and take it off, here are positioned the flash tube unities. Solve in the insulating screw joint three inlet cables. In the connection, uncouple two recessed head screws located on the fixing element of the tube unity and pull out the complete tube unity. The installation occurs in reverse order.

⚠ **Advice:** Do not touch the tubes with the bare hand.

## Care and maintenance

Disconnect the unit from mains supply before executing care and maintenance! Cleaning is effected with a mild cleaning liquid. Clean the glass plate of the polymerisation chamber regularly, a polluted

glass negatively influence the power of the unit. The glass plate can be cleaned with Isopropanole. The used cleaning liquids must not get into the unit.

## General Notes

Repairs on the FotoDent® flash are only to be executed by experts. Disconnect the unit from mains supply at any rate. Dreve Dentamid GmbH is only responsible for the safety, reliability and performance of the unit, if

- the unit is exclusively used for the corresponding application.
- operation is only carried out by accordingly trained personnel.

- the unit is used in accordance with these working instructions.
- extensions, new adjustments, changes or repairs are executed by persons authorised by the company.

As we permanently develop our products further, we reserve the right for technical changes.

## Guarantee

We grant a two year's guarantee for the FotoDent® flash, starting from the date of delivery. The guarantee obligation is only valid for material or production defects.

It does not refer to natural wear and tear nor to damage occurring after the transfer of risk due to improper or unsuitable handling, excessive stressing, unsuitable operating material, and chemical, electro-chemical or electrical influences of a nature not provided for according to the contract. On improper alterations or repairs or maintenance or breach of seals carried out by the purchaser or third parties the guarantee obligations of the supplier and his liability for consequences caused thereby are rendered void.

### Liability

The right of the purchaser to bring actions arising out of defects shall in all cases be in lapse one year after the date of delivery. The warranty obligation does not refer to natural wear and tear or to damage occurring after the transfer of risk due to improper or unsuitable handling, excessive stressing, unsuitable operating material, and chemical, electrochemical or electrical influences of a nature not provided for according to the contract. On improper alterations, repairs or maintenance or breach of seals carried out by the purchaser or third parties our warranty obligations and liability for consequences caused thereby are rendered void.

# FotoDent® flash

## Trouble shooting

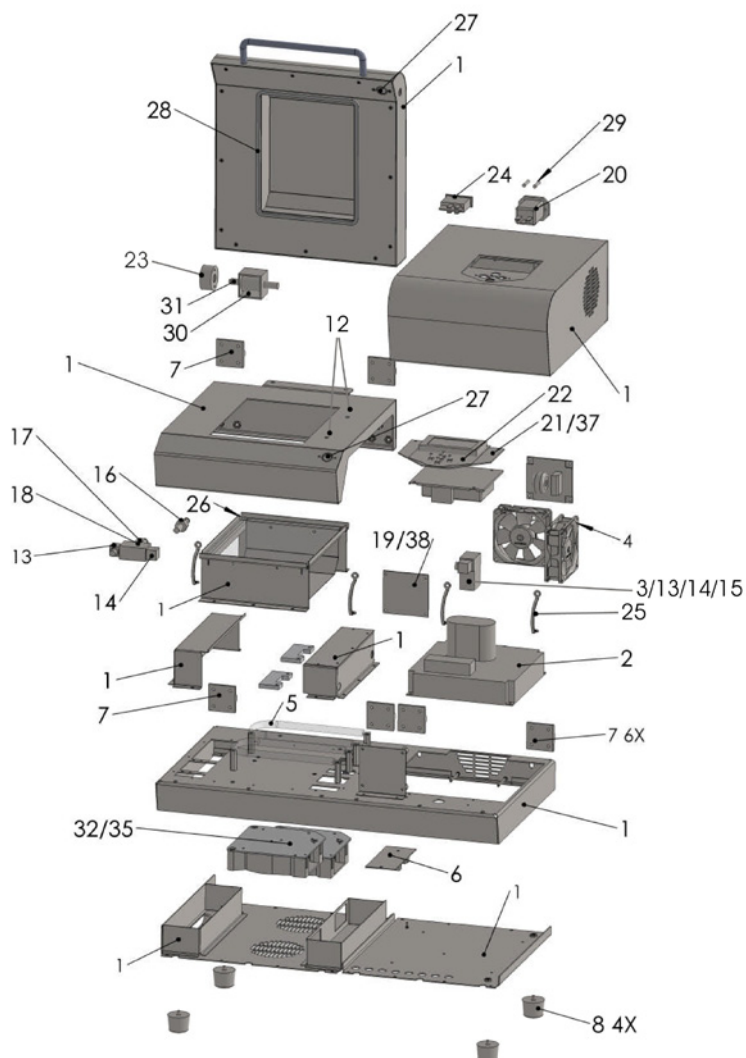
| <b>Problem</b>   | <b>Cause</b>   | <b>Aid</b>                                 |
|--|--|--|
| Unit does not work                                       | Power supply not properly connected  | Check power supply                         |
|  | Main switch off  | Turn main switch on                        |
|  | Micro-fuse defect  | Check micro-fuse and exchange if necessary |
|  | Main switch defect   | Exchange main switch                       |
| Main switch is turned on, electronic timer does not work | Power board is defect  | Exchange power board                       |
|  | Connection cable between power circuit board and control circuit board has loosened itself | Plug in connection cable correctly         |
| Electronic timer works, light tubes do not flashes       | Lid is not closed  | Close lid                                  |
|  | Flash tubes defect, electronic timer shows:<br>Flash bulbs defect                          | Exchange flash tubes                       |
|  | Flashing board defect  | Exchange flashing board                    |
|  | Relay board defect   | Exchange relay board                       |
|  | Ignition transmitter board defect  | Exchange ignition transmitter board        |
| Electronic timer works, program cannot be started        | Lid is not closed  | Close lid                                  |
|  | Lid switch is defect   | Exchange of lid switch                     |

## Trouble shooting

| Problem                 | Cause   | Aid   |
|-------------------------|---|---|
| Objects become too warm | Filter pad in front of the fans are polluted                | Clean filter pads                                       |
|                         | The cooling temperature in the service menu is set too high | Reset the cooling temperature in the service menu       |
|                         | Relay board defect  | Exchange relay board                                    |
| Parts are tacky         | Fan defect  | Exchange fan  |
|                         | Gas bottle is empty   | Exchange gas bottle                                     |
|                         | Pressure reducing valve is closed                           | Open pressure reducing valve                            |
|                         | Lid sealings are defect                                     | Renew lid sealing                                       |
|                         | Value at pressure reducing valve not set to 5 bar           | Set value at pressure reducing valve correctly to 5 bar |
|                         | Needle regulating valve was set different                   | Set needle regulation valve correctly                   |
|                         | Needle regulating valve is defect                           | Exchange of needle regulating valve                     |
|                         | Magnet valve is defect                                      | Exchange magnet valve                                   |
| Lid cannot be locked    | Program has to be started before locking lid                | Start program   |
|                         | Electro magnets are defect                                  | Exchange electro magnets                                |
|                         | Relay board defect  | Exchange relay board                                    |

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## Declaration of conformity

Low Voltage Directive (2006/95/EG).

Herewith we declare that the machine described below in its conception and design and in the shape delivered by us is in accordance with the fundamental requirements for safety and health as prescribed in the applicable EC directives.

In the case of any change or modification of the machine, not authorised by us, this declaration becomes invalid.

|                                     |                                    |
|-------------------------------------|------------------------------------|
| <b>Denomination of the machine:</b> | FotoDent® flash<br>DD35200/DD35201 |
| <b>Type of machine:</b>             | Light polymerisation unit          |
| <b>Applicable EC directives:</b>    |                                    |
| 2006/95/EG                          | Low Voltage Directive              |
| 2004/108/EG                         | EMC Directive                      |

### Applied harmonised standards:

#### 2006/95/EG

EN 61010-1:2010

#### Low Voltage Directive

Safety requirements for electrical equipment for measurement, control and laboratory use – part 1: general requirements (IEC 61010-1:2010)

#### 2004/108/EG

EN 61326-1:2013

#### EMC Directive

Electrical equipment for measurement, control and laboratory use – EMC requirements – Part 1: General requirements (IEC 61326-1:2005); German version EN 61326-1:2006

Emission according to living area, business and industrial undertakings as well as small enterprises. Interference stability according to industrial area.

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Dreve Dentamid GmbH · Max-Planck-Straße 31 · 59423 Unna/Germany