Palavit®55 VS

- DE Gebrauchsanweisung
GB Instructions for use
FR Mode d'emploi
ES Instrucciones de uso
IT Istruzioni per l'uso
PT Instruções de uso
NL Gebruiksaanwijzing
SE Bruksanvisning
DK Brugervejledning
GR Οδηγίες χρήσης
RU Инструкция по применению

GB Instructions for use Palavit®55 VS

Medical Device - for use by dental health care professionals only.
Intended purpose: This dental material is suitable for patients requiring dental treatment for the following indications with consideration of the contraindications.
Product description: Palavit 55 VS is a selfcuring resin for provisionals and other universal applications in fixed and removable prosthetics.
Composition: Methacrylate-Methacrylate-Copolymer; Dibenzoyl peroxide.
Indications: Repair of facings, Repair of methacrylate-crown and bridge veneers, Luting of laboratory fabricated facings (on metal frames), Fabrication of long-term temporaries using denture teeth, Attachment of denture teeth to RPD frameworks, Custom modification of denture teeth, Fabrication of replacement facings.
Contraindications: Do not use: for intraoral application if a dry field is unattainable or the material cannot be processed properly.
Preparation: Mixing Palavit 55 VS.
Attachments: Attachments for use with the Palavit 55 VS.
Warnings: Avoid contact with the skin.
Disposal: Dispose of contents or partially emptied packaging in the household waste or allow them to enter the sewage system.

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