Palaseal®/Palaclean

CE 0044

Palaseal®/Palaclean

- Ⓓ Gebrauchsinformation
- ⒼⒷ User Instructions
- Ⓕ Mode d'emploi
- Ⓔ Modo de empleo
- Ⓘ Istruzioni di impiego
- ⒫ Instruções de uso
- ⒹⒻ Gebruiksaanwijzing
- Ⓐ Bruksanvisning
- ⒹⒶ Brugsanvisning
- ⒼⒾ Οδηγίες χρήσεως
- ⒹⒶⒶ Информация по использованию

**Heraeus
Kulzer**

Heraeus

Heraeus Kulzer GmbH
Grüner Weg 11
63450 Hanau (Germany)
info.lab@heraeus.com
www.heraeus-kulzer.com

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Gebrauchsinformation Palaseal®/Palaclean

Palaseal

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User Instructions Palaseal®/Palaclean

Palaseal

Photocuring lacquer for sealing of dentures and temporary crowns and bridges based on PMMA-resin.

Palaclean

Brush cleaner

Area of application

Palaseal

- Surface sealing of dentures
- Surface sealing of dentures after finishing
- Surface sealing of temporary crowns and bridges

Palaclean

PMMA-based materials are removed from the brush. The brushes can be used repeatedly.

Material

Palaseal is a photocuring denture lacquer which gives a new or repaired denture a permanent smooth surface. The transparent lacquer is not changing the colour of the denture. The smooth surface – which can be attained with Palaseal on denture parts – preclude irritation of the tongue and the mucous membrane. Discoloration is prevented. Deposit of food residue is reduced. Hygienic conditions during the entire usage of a denture are attained.

Special features

- High surface hardness and abrasion resistancy
- Solvent resistancy against denture cleaners
- Good adhesion at denture material
- Smooth surface

Application

The handling of Palaseal is simple. Sandpaper the denture and prepolish with pumice. Do not use greasy or oil cleaning solutions. Dry surface with oilfree air and apply Palaseal with a soft brush in an even thin layer in one direction only. Do not brush a second time. Avoid air entrapments. Polymerize after 20 s for 90 s in the UniXS® or Dentacolor®XS unit. Finishing is not necessary since a smooth cured surface is the result.

Palaclean is designed for cleaning brushes. Soak the brushes in Palaclean for a few minutes, rinse them briefly in clean water and then dry them with a clean cloth.

Attention!

Curing only in Heraflash-, UniXS or Dentacolor XS unit.

The denture must be positioned in the teflon container to allow the complete coated surface to be irradiated. If shadow areas are available (muco-labial fold, interdental areas), a second polymerization of 90 s is required. The denture must then be positioned to allow irradiation of the previous shadow areas.

Danger advice

> *Palaseal*

Product contains: methyl methacrylate
Highly flammable. *Irritating to respiratory system and skin. Risk of serious damage to eyes.* May cause sensitisation by skin contact.

Keep container in a well-ventilated place. Keep away from sources of ignition – No smoking. Avoid contact with skin and eyes. In case of contact with eyes, rinse immediately with plenty of water and seek medical advice. After contact with skin, wash immediately with plenty of water. Wear suitable gloves and eye/face protection.

Palaclean

Highly flammable. Irritating to eyes. Repeated exposure may cause skin dryness or cracking. Vapours may cause drowsiness and dizziness. Keep out of the reach of children. Keep away from sources of ignition – No smoking. In case of contact with eyes, rinse immediately with plenty of water and seek medical advice. Take precautionary measures against static discharges.

Storage advice

Do not store above 25 °C. Avoid direct exposure to sunshine, keep container always closed. Product shall not be used after expiry date. Keep out of reach of children.

Delivery units

Palaseal denture lacquer,
2 x 15 ml net
Palaclean brush cleaner,
45 ml net
Accessories

Refills

Palaseal denture lacquer,
2 x 15 ml net
Palaclean brush cleaner,
45 ml net

® = Registered trademark of
Heraeus Kulzer GmbH