(DE) Gebrauchsanweisung	2	(HU) Használati utasítás	54
(GB) Instructions for use	6	(LV) Lietošanas instrukcija	58
(FR) Mode d'emploi	10	(LT) Naudojimo instrukcija	62
(ES) Instrucciones de uso	14	(PL) Instrukcja obsługi	66
(IT) Istruzioni per l'uso	18	(EE) Kasutusjuhend	70
(PT) Instruções de uso	22	(HR) Upute za uporabu	74
(NL) Gebruiksaanwijzing	26	(RO) Instrucțiuni de utilizare	78
(SE) Bruksanvisning	30	(TR) Kullanma talimatı	82
(DK) Brugervejledning	34	(KR) 사용 설명서	86
(NO) Bruksanvisning	38	(BG) Инструкции за употреба ..	90
(FI) Käyttöohjeet	42	(ID) Petunjuk penggunaan	94
(GR) Οδηγίες χρήσης	46	(SI) Navodila za uporabo	98
(CZ) Návod k použití	50	(RS) Uputstva za upotrebu	102

Signum ceramic bond

Silan-Haftvermittler für Silikatkeramik

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Signum ceramic bond I: Isopropanol, Aceton, Silan, Säure, Monomere, Initiatoren, Stabilisatoren.

Signum ceramic bond II: Silan, Initiatoren, Stabilisatoren, Monomere, Kieselsäure.

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Stand: 2016-01

Signum ceramic bond

Silane coupling agent for silicate ceramic

Components:

Signum ceramic bond I: Isopropanol, acetone, silane, acids, monomers, initiators, stabilizers.

Signum ceramic bond II: Silane, initiators, stabilizers, monomers, silicic acid.

Area of application:

Surface conditioning of silicate ceramic, repair/restoration of ceramic defects with light-curing veneering or filling composites. Can be used intraorally or extraorally.

Contraindications:

- If safe isolation is not guaranteed during intraoral use.
- In the case of known allergy to one of the components.

Dispensing and method of application:

Dispense a few drops each of Signum ceramic bond I and II separately according to requirements in the respective mixing wells and apply as described in section "Technique".

Recommended light-curing repair materials:

- Charisma/Charisma opal
- Venus/flow
- Venus diamond
- Signum ceramis
- Signum composite

Technique

1. Repairing a silicate ceramic without exposed metal framework

Isolate the treatment site. We recommend placing a rubber dam.

- Prepare the ceramic fracture surface (if required) with a diamond cutter under water cooling to remove jagged edges and cracks and to roughen the surface to be conditioned.
- Dry the surface thoroughly using oil-free air.
- Prepare the ceramic surface with a K1 abrasive rotary instrument without applying pressure and at 6,000 to 10,000 rpm; the K1 abrasive should only be used once for reasons of hygiene; there may also be a reduction in the abrasive and cleaning performance. The K1 abrasive should only be used on ceramic surfaces. A diamond rotary instrument (40–60 µm) can also be used for proximal areas that are difficult to access.
- Remove the grinding dust with a short-hair brush or oil-free compressed air, keep dry.
- Apply a thin layer of Signum ceramic bond I to the dust-free ceramic surface using a new brush or microbrush and allow it to dry for approx. 10sec.

- Apply a thin layer of Signum ceramic bond II to the silanized surface and rub in for approx. 20–30sec. with a brush or microbrush. Remove excess – if necessary, using oil-free air or dab it off – no light curing required!
- **Composite build-up:** Layer thickness < 2 mm, light cure with a dental lamp (e.g. Translux®): Kulzer composite 40sec., other products according to the instructions for use. In the laboratory: Cure the composite with HiLite power 180sec. final cure.
Tip: Applying flowable composite (e.g. Venus flow) as a base followed by intermediate curing facilitates adapting the paste material to the ceramic surface.
- **Surface preparation:** Customizing the surface, smoothing the junctions: finishing diamonds, flexible discs.
- **Polishing:** Silicone polishers and felt discs with diamond paste

Storage

Once the container has been opened, we recommend storage in a refrigerator 8°C (46°F). Signum ceramic bond should not be used after the expiry date. Always store the container closed and protected from the light. Avoid direct sunlight.

Hazard/Safety information

Signum ceramic bond I

Highly flammable liquid and vapour. Causes serious eye irritation. May cause drowsiness or dizziness. Keep away from heat/sparks/open flames/hot surfaces. – No smoking. Use explosion-proof electrical/ventilating/lighting/equipment. Avoid breathing dust/fume/gas/mist/

vapours/spray. IF ON SKIN (or hair): Remove/Take off immediately all contaminated clothing. Rinse skin with water/shower. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Store locked up.

Signum ceramic bond II

Causes skin irritation. Causes serious eye irritation. May cause an allergic skin reaction. Avoid breathing dust/fume/gas/mist/vapours/spray. Wear protective gloves/protective clothing/eye protection/face protection. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Take off contaminated clothing and wash before reuse. If skin irritation or rash occurs: Get medical advice/attention. Safety data sheets and more information are available at our website www.kulzer-dental.com

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Kulzer GmbH
Kulzer, LLC