 **FINOETCH**  
Keramikätzgel

 **03819**

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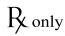
 **03819**

**ENG Operating Instructions**

 Please read carefully before use  
Keep for referral purposes

 Only to be used by qualified  
personnel!

 non-sterile

**1. Intended purpose**  
Hydrofluoric acid gel for etching ceramic restorations (feldspar and glass ceramics) to increase the adhesion between the luting composite and ceramic surface.

**2. Indication**  
Suitable for inlays, onlays, veneers, crowns and bridges.  
Only for extraoral use!

2.1 Contraindication  
 Not suitable for etching hard tooth substance or for intraoral use!

**3. Fields of application**  
For processing in the dental laboratory according to the intended use.

**4. Safety precautions**

 Danger

H310 Fatal in contact with skin.  
H331 Toxic if inhaled.

H314 Causes severe skin burns and eye damage.

4.1 Prevention  
P261 Avoid breathing dust/fume/gas/mist/vapours/spray.  
P280 Wear protective gloves/protective clothing/eye protection/face protection.  
P262 Do not get in eyes, on skin, or on clothing.

4.2 Response  
P301+P310 IF SWALLOWED: Immediately call a POISON CENTER or doctor/physician.  
P305+P351+P338 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.  
P361 Take off immediately all contaminated clothing.  
P302 + P350 IF ON SKIN: Gently wash with plenty of soap and water.  
P312 Call a POISON CENTER or doctor/physician if you feel unwell.

**5. Processing instructions**  
Apply ceramic etching gel to the ceramic surface with an application instrument (disposable brush, plastic stick).  
An etching time of 60 seconds is recommended.  
Then clean the ceramic surface thoroughly and fully remove etching gel residues.  
Cleaning is performed by thoroughly rinsing (min. 20 seconds; do not brush off!) and drying with oil and water-free air.  
Until insertion, the etched area must not be touched or contaminated with saliva.  
If this could not be avoided, repeat the etching process for 10 seconds.  
After renewed cleaning and drying, continue according to the instructions for use of the luting medium.

**6. Storage**  
P233 Keep container tightly closed.  
P411 Storage temperature from °C 10 to °C 25.

**7. Technical data**  
Etching time: min. 60 s; etching time after contamination: 10 s; cleaning time: 20 s; colour: red.

7.1 Composition  
Contains < 5% hydrofluoric acid.

**8. Delivery forms**  
FINOETCH Ceramic etching gel  
5 ml 10046

**9. Guarantee**  
Our technical recommendations of application are based on our own experiences and tests and should only be regarded as guidelines. It rests with the skills and experience of the user to verify that the products supplied by us are suitable for the intended procedures. Our products are undergoing a continuous further development. We reserve the right of changes in construction and composition. It is understood that we guarantee the impeccable quality of our products.

**10. Disposal**  
P501 Dispose of contents/container in accordance with local/regional/national/ international regulations.  
10.1 EWC waste code product  
060103 WASTES FROM INORGANIC CHEMICAL PROCESSES; wastes from the manufacture, formulation, supply and use (MFSU) of acids; hydrofluoric acid; hazardous waste.

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