# Monobond® Plus

## Käyttöohjeet Bruksanvisning Productinformatie Οδηγίες Χρήσεως Kullanım bilgileri Инструкция по применению Instrukcja stosowania

- Yleissidosaine, jolla saadaan aikaan metallin, lasi-/oksidkeramiain ja resiniin välinen sidostus.
- Universalprimer mellom metall, glass-/oksidkeramik og plast
- Universele primer voor verbindingen tussen metaal, glas-/oxidekeramiek en kunststof
- Primer γενικής χρήσης για δεσμό μεταξύ μετάλλου, υαλοκεραμικού / μεταλλοκεραμικού και σύνθετης ρητίνης
- Metal, cam/oksid seramikler ve reçine arasında bir bağ oluşmasına aracılık yapan universal primer
- Универсальный праймер между металлом, стеклокерамикой и пластмассой
- Uniwersalny primer, zwiększający siłę połączenia pomiędzy metalem, ceramiką szklaną i tlenkową oraz cementem kompozytowym.

## Instructions for Use Gebrauchsinformation Mode d'emploi Istruzioni d'uso Instrucciones de uso Instruções de Uso Bruksanvisning Brugsanvisning

- Universal primer mediating a bond between metal, glass/oxide ceramics and resin.
- Universalprimer zwischen Metall, Glas-/Oxidkeramik und Kunststoff
- Primer universel permettant une liaison avec le métal, la vitrocéramique, la céramique cristalline et la résine.
- Primer universale fra metallo, ceramica a base di vetro/ossido e composito
- Primer Universal mediador de adhesión entre metal, cerámicas vidrio/oxi y resina
- Primer universal mediador para ligação entre metal, resina e cerâmicas vítreas e de óxidos
- Universalprimer som förmedlar en bindning mellan metall, glas-/oxidkeramer och kompositmaterial
- Universalprimer mellem metal, glas-/oxidkeramik og plastmaterialer

For dental use only.  
Caution: U.S. Federal law restricts this device to sale by or on the order of a licensed dentist.

Made in Liechtenstein  
Ivoclar Vivadent AG,  
FL-9494 Schaan/Liechtenstein

CE 0123

ivoclar  
vivadent®  
clinical

6.6277/0610WE3EH

English

## Instructions for Use

### Description

Monobond® Plus is a universal primer promoting an adhesive bond between luting composites (particularly the Variolink and MultiLink product lines) and all indirect restorative materials (glass and oxide ceramics, metal, composites, fiber-reinforced composites).

### Composition

Alcohol solution of silane methacrylate, phosphoric acid methacrylate and sulphide methacrylate.

### Indication

Monobond Plus serves as bonding agent and is used to create a durable chemical bond between luting composites and glass/oxide ceramic, metal, composite and fiber-reinforced composite restorations.

### Contraindications

- Any use not listed in the indications.
- If a patient is known to be allergic to any of the ingredients of Monobond Plus.

### Side effects

To date, no systemic side effects are known.

In rare cases, components of Monobond Plus may lead to sensitization. Monobond Plus should not be used in such cases.

### Interactions

Contamination with saliva, blood or water must absolutely be avoided during the entire procedure.

### Application

#### Adhesive cementation of prosthetic restorations:

- 1 Preparing the restoration
  - 1.1 Glass-ceramic restorations (e.g. IPS Empress®)
- Etch with 5% hydrofluoric acid (e.g. IPS® Ceramic

- Etching Gel) for 60 seconds or according to the instructions of the manufacturer of the restorative material.
- Thoroughly rinse with water spray and dry with oil-free air.
- 1.2 Lithium disilicate glass-ceramic restorations (e.g. IPS e.max® Press, IPS e.max CAD)
  - Etch with 5% hydrofluoric acid (e.g. IPS® Ceramic Etching Gel) for 20 seconds or according to the instructions of the manufacturer of the restorative material.
  - Thoroughly rinse with water spray and dry with oil-free air.
- 1.3 Zirconium oxide (e.g. IPS e.max ZirCAD) or aluminum oxide ceramics
  - Sandblast the restoration surface (sandblasting parameters according to the instructions of the manufacturer of the restorative material).
  - If necessary, clean the restoration in an ultrasonic unit for about 1 minute.
  - Thoroughly rinse with water spray and dry with oil-free air.
  - **IMPORTANT!** In order to create a strong bond, do not clean the oxide ceramic surfaces with phosphoric acid.
- 1.4 Metal or metal-supported restorations
  - Sandblast the restoration surface (sandblasting parameters according to the instructions of the manufacturer of the restorative material) until an even, matt or non-glossy surface has been achieved.
  - If necessary, clean the restoration in an ultrasonic unit for about 1 minute.
  - Thoroughly rinse with water spray and dry with oil-free air.
  - **IMPORTANT!** In order to create a strong bond, do not clean the metal surfaces with phosphoric acid.
- 1.5 Composite or fiber-reinforced composite restorations
  - Sandblast the restoration surface (according to the instructions of the manufacturer of the restorative material).
  - If necessary, clean the restoration in an ultrasonic unit for about 1 minute.

- Thoroughly rinse with water spray and dry with oil-free air.
- 2 Apply a thin coat of Monobond Plus with a brush or a Microbrush to the pre-treated surfaces. Allow the material to react for 60 seconds. Subsequently, disperse any remaining excess with a strong stream of air.
- 3 Apply the luting composite (e.g. Variolink®II, Variolink®Veneer or Multilink Automix) according to the respective Instructions for Use and insert the restoration.

### Special notes

- Contamination with saliva, blood or water must absolutely be avoided during the entire procedure. In case of contamination, the entire treatment of the restoration is to be repeated (see section 1).
- If the primer dries out on the surface of the restoration before blasting with a strong stream of air, the surface needs to be rinsed with pure alcohol and blown with air.
- **IMPORTANT!** In order to create a strong bond, do not clean the zirconium oxide surfaces with phosphoric acid.
- Do not insert the brush into the Monobond Plus bottle. This may lead to contamination.
- Carefully close the Monobond Plus bottle immediately after use to prevent evaporation of the solvent.

### Warning

- The product contains ethanol. Keep away from ignition sources.
- Monobond Plus is an irritant.
- Avoid contact with the skin and mucous membrane. If the material should accidentally come in contact with the skin, immediately wash with copious amounts of water.
- In rare cases, contact with the skin may lead to sensitization towards the ingredients.
- Commercial medical gloves do not provide protection against the sensitizing effect of methacrylates.

## Storage

- Monobond Plus should never be used after the expiration date.
- Storage temperature: 2–28 °C / 36–82 °F.
- Shelf life: see expiry date on label.

**Keep out of the reach of children.**

**For use in dentistry only.**

**Date information prepared:** 06/2010, Rev. 2

## Manufacturer

Ivoclar Vivadent AG

FL-9494 Schaan / Liechtenstein

The material has been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions for Use or the stipulated area of application. The user is responsible for testing the products for their suitability and use for any purpose not explicitly stated in the Instructions for Use.

**MSDS available online at [www.ivoclarvivadent.com](http://www.ivoclarvivadent.com)**

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**Ivoclar Vivadent Limited**
**Ivoclar Vivadent, Inc.**