# SHOFU Porcelain Primer

<b>EN</b> Instructions for use	<b>DE</b> Gebrauchsanleitung	<b>FR</b> Mode d'emploi	<b>NL</b> Gebruiksaanwijzing
<b>ES</b> Instrucciones de uso	<b>IT</b> Istruzioni per l'uso	<b>RU</b> Инструкция по применению	<b>PL</b> Instrukcja obsługi
<b>RO</b> Instrucțiuni de utilizare	<b>SV</b> Bruksanvisning	<b>EL</b> Οδηγίες χρήσης	<b>PT</b> Instruções de uso

<b>Manufacturer</b> <div>SHOFU INC.<div>11 Kamitakamatsu-cho, Fukuine, Higashiyama-ku, Kyoto 605-0983, Japan</div>www.shofu.com</div>	<b>EC REP</b> <div>SHOFU DENTAL GmbH Am Brull 17, 40878 Ratingen, Germany</div>
---------------------------------------------------------------------------------------------------------------------------------------	-------------------------------------------------------------------------------------

<b>SHOFU DENTAL CORPORATION</b> <div>1225 Stone Drive, San Marcos, California 92078-4059, USA</div>	<b>SHOFU DENTAL ASIA-PACIFIC PTE.LTD.</b> <div>10 Science Park Road, #03-12, The Alpha, Science Park II, Singapore 117684</div>
-----------------------------------------------------------------------------------------------------	---------------------------------------------------------------------------------------------------------------------------------

<b>03:2019-03</b>	<b>71833</b>
-------------------	--------------

<b>0123</b>
-------------

<b>EN</b>	<b>READ CAREFULLY BEFORE USE</b>
-----------	----------------------------------

## Primer for bonding porcelain or composite

### PRODUCT DESCRIPTION

SHOFU Porcelain Primer is a priming agent designed to treat the surfaces of porcelain or composite restorations to enhance bond with adhesive resin cement. SHOFU Porcelain Primer can be used intraorally and extraorally.

### INDICATIONS FOR USE

Bonding porcelain or composite restorations with adhesive resin cement

### PRECAUTIONS AND WARNINGS

- Do not use this product on patients with known allergies to this material and/or methacrylate monomer
- Operators with known allergies to this material and/or methacrylate monomer must not use this product.
- If any inflammation or other allergic reactions occur on either the patient or operator, immediately discontinue use and seek medical advice.
- Use in well ventilated areas to avoid inhalation of ethanol (several appropriate ventilations per hour are recommended). If inhalation causes nausea or illness, take rest and keep warm in fresh air.
- Avoid any source of ignition since this product is flammable.
- Use protective gloves and glasses to avoid sensitization to this product. In case of accidental contact with oral soft tissue or skin, immediately blot with alcohol moistened cotton ball, and rinse with plenty of water. In case of contact with eyes, immediately flush the eyes with plenty of water and seek medical advice.
- Follow the instructions for use of each material, instrument or equipment to be used in conjunction with this product.
- Do not use this product for any purposes other than specifically outlined in these instructions for use.
- Use this product within the expiration date indicated on the package or container. (Example ☞ YYYY-MM-DD → Year-Month-Date of the expiration date)
- This product is intendeded for use by dental professionals only.

### DIRECTIONS FOR USE

1. Pretreatment of the restoration surface

- Intraoral use

Roughen the adhesive surface of restoration using a dental abrasive (eg. diamond bur, etc.) rinse with water and dry with clean air. Additional etching treatment with Phosphoric acid etchant (concentration of 30-40%) enhances adhesive effect.
- Extraoral use

Sandblast the adhesive surface of restoration with 50-100 µm aluminum oxide at approx. 0.1-0.2 MPa/1-2 bar. Then, rinse with water and dry with clean air.
- Application of SHOFU Porcelain Primer

Dispense adequate amount of SHOFU Porcelain Primer on a dish. Apply to the entire adhesive surface in one layer with a disposable brush. Leave it undisturbed for 10 sec. and dry naturally.
- Seating/Repairing restorations

Follow adhesive resin cement manufacturer’s instructions for use.

### (Notes)

- Do not mix with any other primers or bonding systems.
- Tightly close the cap after each use as this product contains volatile components. Use all the dispensed material promptly.
- Use saliva ejector or suction during the procedure.
- Avoid contamination of the restoration surface to be bonded. If the surface is contaminated by saliva or blood, thoroughly clean the surface with an alcohol moistened cotton ball. Then, dry and reapply the material.
- Use all the dispensed material on only one patient.
- Do not reuse disposable brushes to avoid cross-contamination.

### COMPOSITION

Ethanol, Silane coupling agent etc.

### STORAGE

Store at room temperature (1-30 °C/34-86 °F). Keep away from direct sunlight and any source of ignition.

**CAUTION: US Federal law restricts this device to sale by or on the order of a dental professional.**

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