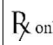
 **FINOFRAME ZR XHT**
Färbelösung

REF 0571/0572/0573/0574

CE 0483

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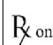
CE 0483

ENG **Instructions for use**

 Please read carefully before use
Keep for referral purposes

 Only to be used by qualified
personnel!

1. Fields of application
Water-based. In 32 different shades for staining ZrO₂ ceramics prior to sintering. The liquid stains the frame completely. As a consequence, this will not result in bright spots when separating sinter drops, connecting bars or when re-milling telescopes. Can be used universally with all conventional ZrO₂ veneer ceramics.

2. Safety precautions

 Note:
Ventilate workstation adequately during processing and drying.
H315 Causes skin irritation.
H335 May cause respiratory irritation.

2.1 Prevention
P261 Avoid breathing dust.
P270 Do not eat, drink or smoke when using this product.
P280 Wear protective gloves/eye protection.

2.2 Response
P305+P351+P338 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.
P302 + P350 IF ON SKIN: Gently wash with plenty of soap and water.
P301+P330+P331 IF SWALLOWED: rinse mouth. Do NOT induce vomiting.
P304+P341 IF INHALED: If breathing is difficult, remove victim to fresh air and keep at rest in a position comfortable for breathing.
P314 Get medical advice/attention if you feel unwell.
(use safety data sheet)

3. Processing instructions
3.1 Frame preparation
Prior to staining, the zirconium frame will be finished completely, be free of dust and be completely dry.
To give the highest level of transparency, clean the restoration in an ultrasonic cleaner with distilled water. Swirl the restoration in the distilled water of the ultrasonic cleaner for 5 to 10 seconds using plastic tweezers until no more dust separates from the restoration. Remove excess water from the restoration with oil-free compressed air or cellulose or cotton swabs

3.2. Processing
Dry restoration under infrared lamp or in ceramic furnace.
Do not touch the objects with moist or oily hands as the colouring liquid cannot penetrate into the zirconium frame otherwise. Shake bottle with colouring liquid thoroughly before use.
Pick up the stain with a thin brush directly from the respective bottle. Before using a new bottle please rinse brush thoroughly and dry with some cellulose.
Initially apply the blocker to the areas where the shade of the zircon should stay unchanged. The blocker prevents the penetration of other stains into the restoration.
Then apply the colouring liquids for the cervical areas, the cusps and incisal edges and incisal areas.
Subsequently apply occlusal stains respectively effect stains and finally dentin stains with a thin brush. The application of the dentin stain from the inside of the crown suggests the illusion of a dentin core.

 Note:
Dilute liquid with distilled water to reduce the colour intensity and to brighten the colour shade.

3.3 Drying/sintering
Dry the frame for 10 minutes on a firing tray under an infrared lamp. Then sinter as usual.
If necessary, adapt the frame to the working model by employing a wet grinding process with diamonded abrasives. Do not use sintered diamonds, corundum stones or carbide cutters.

3.4 Firing the ceramics
All conventional ZrO₂ veneer ceramics with a CTE of 10-10,6 x 10⁻⁶K⁻¹ can be used. After sintering, clean the zirconium framework with a steam cleaner and envelop the surface of the frame completely with ceramic. Observe the instructions for use for the ceramic material.

4. Storage
Keep the colouring liquid sealed securely and out of reach of children at a temperature of 4-10 °C (refrigerator).

5. Physical data
Density 1 g/cm³; water solubility 100 %.

5.1 Composition
Polyethylene glycol in aqueous solution and small quantities of ferric(III)-chloride, vanadium(III)-chloride, chromium(III)-chloride.

5.2 Hazardous ingredients
No further data.

6. Delivery forms
See supplement.

7. Guarantee
Our technical recommendations of application are based on our own experiences and tests and should only be regarded as guidelines. It rests with the skills and experience of the user to verify that the products supplied by us are suitable for the intended procedures. Our products are undergoing a continuous further development. We reserve the right of changes in construction and composition. It is understood that we guarantee the impeccable quality of our products.

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