FINOPUR Gipsabscheider

REF 10386

DEU Gebrauchsinformation

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REF 10386

ENG User Information

 Please read carefully before use
Keep for referral purposes

1. Intended purpose

4-chamber cleaning system made of easy-care polypropylene.

2. Indication

For the treatment of waste water containing plaster in dentistry.

2.1 Contraindication

Not known.

3. Fields of application

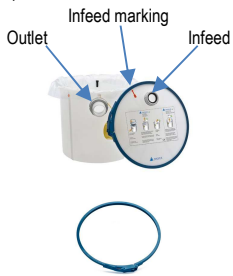
For processing in the dental technology according to the intended use.

4. Safety precautions

No particular hazards worth mentioning.

5. Processing instructions

5.1 Components

5.2 Commissioning

The unit must only be used for the intended purpose described in the scope/field of application.

Before first operation check the item for transport damages. Claim any transport damages immediately with the supplier.

Connect plaster separator with the inlet (Ø 47/40 mm) via a feeder hose (Ø 28 mm) to the wash basin and with the outlet (50 mm/53 mm) to the drain.

Depending on the space available, e.g. under the wash basin, use a straight or 90° angled feeder hose for installation.

Insert the closed edge of the cross-shaped insert of the separator so that it is positioned to the left of the outlet.

Insert the inlet into the separation chamber with the marking.

5.3 Emptying the plaster separator

The transparency of the outer wall of the plaster separator allows a visual check of the sludge level in the separator.

To clean, loosen the connection piece of the outlet and open the sealing cap.

Remove the outlet adapter, seal the foil insert below the opening of the outlet and then dispose of it.

5.4 Flocculant

To allow plaster-free water to flow off, FINOPUR TABS flocculant can be placed in the second chamber, adjacent to the wall of chamber 1, using a clip-on basket.

Replace the tablet after about one week.

6. Storage

No special storage conditions required.

7. Technical data

Capacity 14 l; Overall height 320 mm; Diameter inner ø 295 mm; outer ø 340 mm; Draining height 245 mm; Connection infeed inner ø 39 mm; outer ø 47 mm; Connection outlet outer ø 50 mm; Weight 2,1 kg; Temperature-stable up to max. 80°C/176°F.

8. Delivery forms

FINOPUR plaster separator complete with lid, plastic bag insert and 2 FINOPUR TABS flocculant with clip-on basket
1 piece 10386

9. Accessories

FINOPUR			
plastic bag	25 pieces	10387	
Drain hose, for connections with ø 40 oder 50 mm			
	1000 mm	1 piece	10392
	750 mm	1 piece	10393
	500 mm	1 piece	10394
Feeder hose straight 1 ½ inch, hose ø 28 mm		1 piece	10388
Supply hose 90° 1 ½ inch, hose ø 28 mm		1 piece	10389

10. Guarantee

Our technical recommendations of application are based on our own experiences and tests and should only be regarded as guidelines. It rests with the skills and experience of the user to verify that the products supplied by us are suitable for the intended procedures. Our products are undergoing a continuous further development. We reserve the right of changes in construction and composition. It is understood that we guarantee the impeccable quality of our products.

11. Disposal

P501 Dispose of contents/container in accordance with local/regional/national/ international regulations.

11.1 EWC waste code product
070213 WASTES FROM ORGANIC CHEMICAL PROCESSES; wastes from the MFSU of plastics, synthetic rubber and man-made fibres; waste plastic.

12. Symbols explanation

12.1 Manufacturing

	Item numbers
	Lot code
	Shelf life
	non-sterile

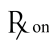
 Unit in the packaging

 Manufacturer

 Date of manufacture

12.2 Application

 Please read carefully before use
Keep for referral purposes

 Only to be used by qualified personnel!

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