# FINOSIL 15/F 15

## Dubliersilikon

REF 14200/14200P/14201/14213

### DEU Gebrauchsinformation

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REF 14200/14200P/14201/14213

ENG User Information

Please read the entire document before using the Duplicating silicone.

### 1. Intended purpose

Addition-crosslinked duplicating silicone for the dental duplicating technique. Practical and to be mixed precisely in a ratio of 1:1. Precise in detail, shrink-free and stable in storage.

### 2. Indication

Suitable for all types of duplication and virtually any material, e.g. plaster, modelling resins and metal.

### 2.1 Contraindication

Not known.

In case of allergic reactions to product components, sensitive individuals cannot be excluded.

### 3. Fields of application

#### 3.1 General

Only for processing in the dental technology according to the intended use.

- In the event of use other than intended, all warranty and liability claims become void!
- Shelf life: Consumables should generally be used up quickly and promptly within a few months after opening the original packaging.

#### 3.2 Target group

The target user groups for Duplicating silicone are:

- Dentists
- Dental technicians

The patient target group for Duplicating silicone includes patients receiving as part of dental treatment.

### 3.3 Cleaning

Duplicating silicone are products intended for single use. They do not require cleaning.

Duplicating silicone are not intended for disinfection.

### 4. Safety precautions

The national legal regulations must be observed during use, in particular:

- the applicable health and safety regulations
- the applicable accident prevention measures

No particular hazards worth mentioning.

Further safety-relevant data is given in the safety data sheet.

### 5. Processing instructions

Shake well before use!

#### 5.1 Preparation

Do not water model. Carefully remove contamination from the surfaces (separating agent, instant adhesives, spacer varnish etc.)

#### 5.2 Dosing

Volume ratio component A catalyst, white : component B base, coloured / 1 : 1

#### 5.3 Duplicating of model

##### 5.3.1 Manually

Place components A and B in mixing cup at a ratio of 1 : 1. Stir well with a spatula until a homogeneous mixture (colour) has been achieved.

Pour duplicating silicone in a thin stream into the flask or duplicating form.

##### 5.3.2 Automatically

Rational and clean dosage, mixing and batching in a duplicating and mixing device (e.g. FINODOSIL duplicating and mixing device, item no. 14290).

Strength of stream is defined by the duplicating and mixing device, distance to flask or duplicating form approx. 20 cm.

### 5.4 Curing

Optimal results are achieved through pressure duplication in a pressure pot. Allow duplicating silicone to cure (vulcanisation) until complete recovery is achieved for approx. 35 min. or 10 min. (FINOSIL F 15) and devest model.

### 5.5 Pour

The negative mould is highly elastic, permanently dimensionally stable and can be cast several times.

Adhere to the reset time of 30 minutes between removing the model and repeated casting to ensure complete recovery.

Spray the negative mould with silicone wetting agent (e.g. FINO SPANN-EX silicone wetting agent, item no. 14225) for wetting the surface and preventing the formation of bubbles.

All dental plasters, modelling resins and investment materials are suitable for casting. For pressure duplication, fabricate model at same pressure as negative mould.

### 6. Storage

P404 Store in a closed container.

P411 Store at temperatures not exceeding 25 °C.

### 7. Physical data

FINOSIL 15 duplicating silicone normal-curing:

Mixing time manually 45 s; mixing time under vacuum approx. 30 s; processing time 6 ± 1 min.; demoulding after 30 ± 5 min.; final hardness 15 Shore A; mixing ratio, component A : B = 1 : 1; A = catalyst, white; B = base, yellow.

FINOSIL F 15 duplicating silicone fast-curing:

Mixing time manually 45 s; mixing time under vacuum approx. 30 s; processing time 3 min.; demoulding after 10 min.; final hardness 15 Shore A; mixing ratio, component A : B = 1 : 1; A = catalyst, white; B = base, orange.

### 7.1 Composition

Vinyl-polysiloxane-based addition curing duplicating silicone.

### 8. Delivery forms

Packaging units for Duplicating silicone are:

- Duplicating Silicone
- User Information

The scope of delivery of each individual product consists of:

FINOSIL 15 Duplicating Silicone	2 x 682 ml	14200
	2 x 5172 ml	14201
	2 x 86 ml	14200P
FINOSIL F 15 Duplicating Silicone	2 x 850 ml	14213

Duplicating Silicone are shown in the illustration:

Duplicating silicone are supplied without accessories.

### 9. Guarantee

Our technical recommendations of application are based on our own experiences and tests and should only be regarded as guidelines. It rests with the skills and experience of the user to verify that the products supplied by us are suitable for the intended procedures. Our products are undergoing a continuous further development. We reserve the right of changes in construction and composition. It is understood that we guarantee the impeccable quality of our products.

### 10. Exclusion of liability

All rights reserved. This document may not be reproduced, transmitted, transferred, distributed, modified, summarised or translated into any other language graphically, electronically or mechanically, including but not limited to photocopying, recording, information storage and processing in systems, without the prior written consent of FINO GmbH. Copies of the system software are not permitted.

### 11. Disposal

P501 Dispose of contents/container in accordance with local/regional/national/ international regulations.

#### 11.1 EWC waste code product

070216 WASTES FROM ORGANIC CHEMICAL PROCESSES; wastes from the MFSU of plastics, synthetic rubber and man-made fibres; waste containing hazardous silicones; hazardous waste.

### 12. Symbols explanation

#### 12.1 Manufacturing

- REF Item numbers
- LOT Lot code
- Shelf life
- non-sterile
- Unit in the packaging
- Manufacturer

#### 12.2 Application

General danger symbol warns of the risk of serious injury in conjunction with the warning words CAUTION, WARNING or DANGER

- Notes
- Important information
- Please read carefully before use
- Keep for referral purposes
- Do not reuse

#### 12.3 Storage

- Temperature limit
- Keep away from direct sunlight
- Protect against moisture
- top - store upright
- Caution fragile

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