 **FINO**
Basis-/Löffelplatten

REF 14975/14977/14978

CE

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CE

ENG **Instructions for use**

 Please read carefully before use
Keep for referral purposes

 Only to be used by qualified
personnel!

1. Intended purpose
For fabricating base plates for bite templates
and bite registrations, individual impression
and function trays, etc. Light-curing.

2. Indication
Base plates for wax prosthesis set-ups.

2.1 Contraindication
Do not use if incompatibilities or allergies to
constituents have been confirmed.

3. Fields of application
Only for processing in the dental laboratory
according to the intended use.

4. Safety precautions

 Warning
H312 Harmful in contact with skin.
H317 May cause an allergic skin reaction.

4.1 Prevention
P261 Avoid breathing dust/fume/gas/mist/
vapours/spray.
P272 Contaminated work clothing should not
be allowed out of the workplace.
P280 Wear protective gloves/protective
clothing/eye protection/face protection.

4.2 Response
P302+P352 IF ON SKIN: Wash with plenty of
water.
P312 Call a POISON CENTER or doc-
tor/physician if you feel unwell.
P333+P313 If skin irritation or rash occurs:
Get medical advice/attention.
P362 Take off contaminated clothing and
wash before reuse.
Use work bench extraction system when
grinding.

5. Processing instructions
5.1 Preparation
Plot the extension options and reduction ne-
cessities for the individual trays on the work
model. Block out undercuts.
When working without a place holder, insu-
late the model immediately after wetting with
a suitable alginate insulation (for example,
FINOSOL, Item no. 11445). Form the FINO
base plate/tray plate true to the model.
When using a place holder, adapt the FINO
base plate/tray plate directly on the place
holder.
Trim protruding parts to the marked tray
edge.
A tray handle can be formed from the excess
material.
Seal pack well after removing material.

5.2 Polymerisation
Polymerisation time with conventional light
curing devices, (e.g. FINOLUX light curing
device, item no. 14954) 5 min. Extend
polymerisation time by 90 s in case of very
thick tray handle.
Remove individual tray carefully from the
model after polymerisation and polymerise
the basal surface again for 5 min.

5.3 Work up
After curing, the FINO base plate/tray plate
can be processed further immediately. Boil
off wax residues with hot water. Recommen-
dation rotary instruments; cross-linked car-
bide milling cutters.

6. Storage
P102 Keep out of reach of children.
P404 Store in a closed container.
P410 Protect from sunlight.
P411 Store at temperatures not exceeding
25 °C.
Do not use beyond expiry date.

7. Physical data
Flexural strength (UVA) ≥ 85 N/mm²;
modulus of elasticity (UVA) ≥ 6000 N/mm²;
shrinkage 2.08 vol.-%;
absorption of photo indicator combination
360-480 nm;
plate thickness tray plates 2.2 ± 0.2 mm;
plate thickness base plates 1.4 ± 0.2 mm.

7.1 Composition
Acrylic resin, photo initiators, stabilisers, col-
our dyes, spherical glass fillers.

8. Delivery forms

FINO base plates pink	50 pcs.	14975
FINO tray plates transparent	50 pcs.	14977
blue	50 pcs.	14978

9. Guarantee
Our technical recommendations of applica-
tion are based on our own experiences and
tests and should only be regarded as guide-
lines. It rests with the skills and experience of
the user to verify that the products supplied
by us are suitable for the intended proce-
dures. Our products are undergoing a contin-
uous further development. We reserve the
right of changes in construction and com-
position. It is understood that we guarantee the
impeccable quality of our products.

10. Disposal
P501 Dispose of contents/container in ac-
cordance with local/regional/national/interna-
tional regulations.

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