



Duplication-form for the production of single-use plaster-stabilizers for model duplication.

For efficiently duplication of plaster-models.

With this silicon form (are available in 2 different sizes) it is possible to produce an unlimited number of single-use plaster-stabilizers. So it is irrelevant how many technicians currently require a flask.

The geometry enables an easy removal of the plaster-stabilizer. At the same time the plaster-stabilizer warrants a continuous and stable position of the silicone after the duplication process.

The model-removal is gentle and simple, wether the models are made of plaster, refractory material or synthetic substances.

Due to the simple reproduction of single-use plaster-stabilizers, they can be easily archived.



Fig 2
Silicone form filling with plaster



Fig 3
removal of the hardend raw plaster-stabilizer.



Fig 4
retention windows are pierced with a pointy instrument.



Fig 5
the base is trimmed until the bottom of the flask is fully removed.



Fig 6
The plaster-stabilizer in position with the Silflex-Flask (DeguDent).



Fig 7
filling the flask with duplication silicon



Fig 8
The finished silicon-form of the master-model stabilized with the plaster-stabilizer.



Fig 9
After resetting to the starting position the duplication is ready for filling with refractory material/plaster/synthetic resin and can be easily labelled and archived.



Infos:

Made by Uwe Gehringer - Dentallabor
Frauenstraße 11
D-80469 München
Tel.: +49 89 260 100 86
Fax: +49 89 260 100 87
uwe@madeby-ug.com

Distribution:

DT&SHOP
Mangelfeld 11-15
D-97708 Bad Bocktät
Tel: +49 9708 909-100
Fax: +49 9708 909-125
info@dt-shop.com
www.dt-shop.com