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11.1 Registrierungsnummer
WEEE-Reg.-Nr.
DE 75542610

Please read the entire document before using the induction heating device.

1. Intended purpose

Modelling aid for use in dentistry.

2. Indication

Modelling aid for soot-free heating of instruments without a flame.

2.2 Contraindication

Not known.

3. Fields of application

For processing in the dental technology according to the intended use.

4. Safety precautions

Do not touch with damp hands.

Avoid splash water.

Do not insert a finger with a metal ring into the opening to avoid burns.

Avoid shock or impact, these can lead to mechanical problems.

5. Operation

5.1 General

Before first operation check the item for transport damages. Claim any transport damages immediately with the supplier.

The unit comes ready-to-use.

Environmental conditions for safe operation:

- Operate the device exclusively in closed rooms (up to a maximum of 2000 metres above sea level)
- Temperature from 5 to 40 °C (41-104°F *)

- Maximum relative humidity 80 % at 31 °C (87.8 °F), linear decrease up to 50 % at 40 °C (104 °F) *)
- At voltage fluctuations in power supply from the mains not greater than 10 % of the nominal value
- Contamination level 2
- Overvoltage category II

*) At 5-30 °C (41-86 °F), the device can be operated at a humidity of up to 80 %.

At temperatures of 31-40 °C (87.8-104 °F) the humidity must decrease proportionately to ensure readiness for use (e.g. at 35 °C (95 °F) = 65 % humidity at 40 °C (104 °F) = 50 % humidity). The device may not be operated at temperatures over 40 °C (104 °F).

5.2 Electrical connection

Check if the voltage information on the type label of the unit matches the voltage of the mains supply. Install power cable in a way that it does not is a tripping hazard. Plug mains plug in.

Do not place the device near heat sources.

Do not block the fan at the bottom.

5.3 Handling

The device is switched on and off with the ON/OFF switch on the rear panel.

The LED lights green in standby mode.

Place the supplied protective cap into the opening and insert a magnetised metal instrument into the centre of the opening.

Do not touch the inside of the opening!

A sensor on the top of the device switches the device into operation when instruments are detected.

The LED lights up red.

The instrument is heated by induction.

In case of overheating, the green LED warns by flashing.

If a malfunction occurs, the LED flashes red and green alternately.

In this case, stop working and wait until the LED returns to green.

If the error message persists, unplug the mains cable and try again after a few minutes.

To avoid malfunctions, do not bring the optical sensor into contact with molten wax.

5.4 Setting sensitivity

When using a thin instrument, the sensitivity of the sensor can be set higher so that the instrument reacts quicker.

Anticlockwise rotation: lower sensitivity.

Clockwise rotation: greater sensitivity.

Turn the screw slowly in the desired direction using a screwdriver until the LED changes from red to green.

5.5 Service and maintenance

Always disconnect the unit from the mains supply before cleaning it or performing any maintenance on it. Do not spray the unit with water, immerse it in water or expose it to splash water.

Do not use chemicals for cleaning.

6. Storage

Store the device at room temperature protected against moisture.

7. Technical data

Power rating: 220/230V, 50/60 Hz; Output: 100 W; Fuse: T2AL, 250V; Dimensions: 9,1 x 20,9 x 8,9 cm; Weight: 530 g

8. Delivery forms

Packaging units for Induction heating device are:

- Induction heating device
- Operating Instructions

The scope of delivery of each individual product consists of:

FINO FLAMELESS Induction heating device 1 pc. 20310

Protective caps 4 pcs. 20315

Induction heating device are shown in the illustration:

9. Accessories

Protective caps 10 pcs. 20315

10. Guarantee

A condition for guarantee claims is the availability of the original sales invoice from the dental retailer. Our products are subject to continuous further development. We therefore retain the right to changes in design and composition. Of course we guarantee the perfect quality of our products.

! Important:

FINO GmbH has prepared these instructions to help users operate the unit and to make it easier to take the unit into operation for the first time and then use it. FINO GmbH cannot be held liable for damages due to improper use or due to modifications made to the device itself. The warranty

will be voided if the unit is used for any purpose other than its proper intended purpose. FINO GmbH has a policy of continuously developing and improving its products. FINO GmbH reserves the right to make changes or improvements to the products described in this document or to the document itself without prior notice. FINO GmbH offers no guarantees for the completeness or correctness of this document. Performances under guarantee do not extend the term of guarantee.

11. Disposal in accordance with EU guideline 2002/96/EU

This equipment may not be disposed in domestic waste! Please contact the manufacturer or your dealer!

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