
ERKOLOC-PRO

Lagerung • storage • stockage

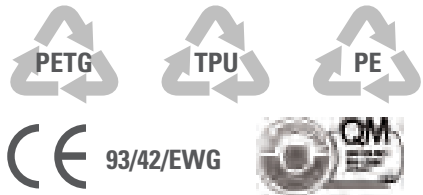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E ERKOLOC-PRO

Chemical characteristics:

PETG, hard side (1,2,3 + 4 mm): copolyester.
 TPU, soft side (1 mm): polyurethane.
 PE, insulating foil: polyethylen
 Harmless to health, BgVV (Federal Health Office)
 listed thermoplasts with tested biocompatibility.
 Insoluble in water, inactive, harmless to ground
 water.

Disposal/Recycling:

Household waste (combustable, low in pollutants
 and high in energy)/general recycling for plastic.

Field of use:

(see brochure „Thermoforming technique“ and
 material card)
 All kinds of comfortable occlusal and bruxism
 splints. Adjustment is possible with the Occluform,
 by grinding or by addition.

Material properties:

Durable, resistant double layer plate, hard/soft. Hard
 layer adjustable by addition using acrylics (Resilit-S,
 817 501). **Attention:** The soft layer does not bond to
 Erkoflex.

Finishing: (see brochure „Thermoforming tech-
 nique“). Finishing set -Quick- (110 875). The insulat-
 ing foil on the soft layer will only be removed after
 thermoforming and finishing. With polish for plastic
 the hard side can be given a high polish.

Plastification:

(for machines without heating time control)
 The hard side of the ERKOLOC-pro plate should be
 heated. Test softness of foil with an instrument. If
 permanent impressions result, thermoform.

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