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Please read the entire document before using the product.

1. Intended purpose

Firing Paste for use in dentistry.

2. Indication

High-temperature stable liquid firing wool for stabilising metal-free ceramic restorations.

Suitable for fixating soldering objects and platinum foils for cast repairs, as well as for repair work on pressed ceramics and ceramic margins.

2.1 Contraindication

Intended for use only in the specific area of application of the dental laboratory.

Any use beyond that described in these instructions is considered to be improper use. The manufacturer shall not be held liable for any resulting damage.

3. Fields of application

For processing in the dental technology according to the intended use.

4. Safety precautions

No particular hazards worth mentioning.

5. Processing instructions

5.1 Stabilisation of objects to be fired

Apply the liquid firing wool directly into the restoration.

Do not fill ceramic crowns completely to avoid excess material on the outer side of the crown.

Then, carefully place the objects to be fired on a holding pin of the firing tray.

No drying time required.

The material hardens completely during the firing process.

5.2 Creating an individualised firing tray

The desired amount of firing paste is applied to the firing tray.

The material can be moulded into the desired shape.

Fix the paste by applying heat from a laboratory burner.

The restoration can now be placed on the individual firing tray.

If necessary, fill the inside of the restoration and also attach a supporting pin.

5.3 Fixation of soldering work

Workpieces that require soldering can be fixed in place with the fireproof material, which thus serves as an alternative to soldering investment material.

5.4 Repair of cast parts

Suitable for fixating platinum foil.

The desired amount is simply applied into the object for soldering.

The paste does not require any drying time and hardens completely when heat is applied.

Then start the soldering process as usual.

For repairs without platinum foil, a sufficient amount of firing paste is applied to the crown.

Then the damaged area is closed with solder.

After hardening, the firing paste is removed again.

5.5 Surface protection

For high-temperature work up to 1925 °C, the firing paste can be applied to the surface requiring protection.

5.6 Removing the firing paste

The hardened material can easily be removed with tweezers or another suitable instrument.

It is also possible to use compressed air.

Residues can be removed without leaving any marks using a steam cleaner or under running water.

6. Storage

Store the closed product in a cool, dry place.

7. Physical data

Temperature stable up to 1300 °C.

8. Delivery forms

FINOBURN FIX
Firing Paste

1 piece 30491

Firing Paste are shown in the illustration:

9. Guarantee

Our technical recommendations of application are based on our own experiences and tests and should only be regarded as guidelines. It rests with the skills and experience of the user to verify that the products supplied by us are suitable for the intended procedures. Our products are undergoing a continuous further development. We reserve the right of changes in construction and composition. It is understood that we guarantee the impeccable quality of our products.

10. Exclusion of liability

All rights reserved. This document may not be reproduced, transmitted, transferred, distributed, modified, summarised or translated into any other language graphically, electronically or mechanically, including but not limited to photocopying, recording, information storage and processing in systems, without the prior written consent of FINO GmbH. Copies of the system software are not permitted.

11. Disposal

P501 Dispose of contents/container in accordance with local/regional/national/international regulations.

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