# FINOPOLY

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Germany

51615

Poliermotor  
Polishing Lathe  
Tour à polir  
Pulidor  
Pulitore  
Polijsmotor  
Leštička  
Polerka

FINO · der feine Unterschied · the fine difference · la fine différence · la fina diferencia · la fine differenza

Gebrauchsinformation · User Information · Mode d'emploi  
Información sobre el uso · Informazioni d'uso  
Gebruiksaanwijzing · Instrukcja stosowania

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# INDEX

<b>1. Fields of application</b>	<b>3</b>
<b>2. Advantages</b>	<b>3</b>
<b>3. Safety precautions</b>	<b>3</b>
<b>4. Hazard information</b>	<b>3</b>
<b>5. Characteristics</b>	<b>3</b>
<b>6. Place of installation</b>	<b>3</b>
<b>7. Commencement of operation</b>	<b>3</b>
<b>8. Operation</b>	<b>4</b>
• Polishing • Removal of polishing spindle	
<b>9. Maintenance and care</b>	<b>4</b>
<b>10. Storage</b>	<b>4</b>
<b>11. Technical data</b>	<b>4</b>
<b>12. Availability</b>	<b>4</b>
• Accessories	
<b>13. Guarantee</b>	<b>4</b>
<b>14. Troubleshooting</b>	<b>5</b>
<b>15. Disposal</b>	<b>5</b>
• Registration number	
<b>Important!</b>	<b>5</b>

## 1. Fields of application

The FINOPOLY Polishing Lathe is a powerful motor, whose construction was specifically designed as a polishing lathe for dental technology.

## 2. Advantages

- Easy operation
- Sturdy construction
- High performance
- Maintenance-free

## 3. Safety precautions

### 3.1. Marking of information

Before you start operating the unit it is positively necessary to read the user information carefully and to store it closely at hand.

This symbol marks safety information, whose non-observance may cause hazards for the unit and its function.

This symbol marks safety information whose non-observance may cause hazards to persons.

This symbol marks tips or information, which may facilitate your work and ensure a safe operation.

## 4. Hazard information

It is recommended to wear protective eyewear and a protective mask whenever the FINOPOLY Polishing Lathe is operated.

If damages at the connecting cable or plug are visible the unit must be put into operation.

The FINOPOLY Polishing Lathe operates with very high rotational speeds. If not used according to specifications or during work, which deviated from normal dental technological applications, the unit is to be regarded as hazardous for the operator and all persons in its immediate neighbourhood.

Never try to block the rotation of the spindles with a foreign body respectively with an object that is not processed or with the hands. This can show to be extremely dangerous for the user as well as for the unit.

## 5. Characteristics

The unit is designed for permanent operation. The shafts are prepared to take standard spindles. The FINOPOLY has a low-noise motor and maintenance-free special ball bearings. The housing has a broad base with rubber feet, which improves stability and supports self-cooling. The high performance of the motor guarantees a stable speed of 1400 respectively 2400 rpm. After a power cut the unit does not start automatically but it has to be switched on again. When the polishing lathe is switched on both shafts start to rotate simultaneously. The design of the unit ensures that brushes do not come in contact with the housing.

## 6. Place of installation

Take care that the installation surface is level and plain. The high own weight and the special rubber feet ensure a stable stand of the lathe even on smooth surfaces. Place the connecting cable in a way that nobody can trip over.

## 7. Commencement of operation

Please check the unit for transportation damages immediately after unpacking. All transportation damages have to be claimed promptly. At first check if the voltage indication on the type label of the unit matches the voltage of your mains supply. Is this the case you can plug the unit into the socket.

## 8. Operation

### 8.1. Installation of polishing tools

The left polishing spindle is marked with a groove. Set the tool on the polishing spindle and screw the spindle manually tight into the tool. You must screw the spindle right or left into the polishing tool depending on the type of spindle (right or left).

For de-mounting it might happen that you have to use a tool because the polishing tool gets tied up during operation.

### 8.2. Polishing

Actuate the main switch. The control light in the switch lights up. If the switch does not catch check the power supply.

The speed selector has three positions:

Vertical position      unit is in stand-by 0 rpm

Left position          low speed 1400 rpm

Right position        high speed 2800 rpm

### 8.3 Removal of polishing spindle

Do never pound on the drive shaft or the polishing tools. Thus the bearing could be damaged.

#### Left polishing spindle

Turn the left de-mounting aid to the right until the polishing spindle comes off the shaft. Subsequently tighten the de-mounting aid again.

#### Right polishing spindle

Turn the right de-mounting aid to the left until the polishing spindle comes off the shaft. Subsequently tighten the de-mounting aid again.

## 9. Maintenance and care

For cleaning and checking of the unit you must always unplug the unit. The FINOPOLY Polishing Lathe is maintenance-free. The bearings of the unit are provided with a long-term lubrication by the manufacturer. Therefore avoid any lubrication of the bearings. This could cause damages. It is recommended to clean the varnished housing regularly. Cleaning agents for metal must not be used. Please remove chemicals and plaster residues immediately. From time to time check the condition of the spindles. The motor has on both sides conically milled shaft ends with the standard cone 9 mm. These must be treated gently. Pounding on the shaft ends can damage the ball bearings and cones.

Shaft ends not in use should be covered with an oil film to prevent corrosion. Before you mount a polishing spindle or disk-carrier the oil film must be removed to achieve a good fixation.

Make sure that the power cable is not damaged.

Repairs of the unit should only be done by FINO or an authorised workshop.

The user has to stand good for damages caused by unauthorized/arbitrary repairs. The unrestricted conditions of delivery and guarantee of FINO apply for this unit.

## 10. Storage

Store the FINOPOLY Polishing Lathe in a dry and clean place.

## 11. Technical data

Voltage:                    230 V, 50/60 Hz

Output:                    400 Watt

Speed:                    1400/2800 rpm

Dimensions (L x W x H): 51 x 15 x 22 cm

Height of shaft:        17 cm

Weight:                    11 kg

## 12. Availability

FINOPOLY Polishing Lathe                    each 51615

FINOPOLY Polishing Spindle right        each 51617

FINOPOLY Polishing Spindle left        each 51618

### 12.1. Accessories

FINO Polishing Trough                    each 51616

## 13. Guarantee

Our technical recommendations of application are based on our own experiences and tests and should only be regarded as guidelines. It rests with the skills and experience of the user to verify that the products supplied by us are suitable for the intended procedures. Our products are undergoing a continuous further development. We reserve the right of changes in construction and composition. It is understood that we guarantee the impeccable quality of our products.

## 14. Troubleshooting

### 14.1. The unit does not work:

- Make sure that the voltage indication on the type label of the unit matches the voltage of your mains supply.
- Make sure that the plug is properly plugged into the socket.
- Check if the fuse of your mains distribution has tripped.

### 14.2. The unit makes abnormal noises:

With a high probability the bearings of the shaft are worn. Contact DT&SHOP or an authorised workshop.

## 15. Disposal in accordance with

### EC Guideline 2002/96/EG

This equipment may not be disposed in domestic waste! Please contact the manufacturer or your dealer!

### 15.1 Registration number

3092617

### Important!

FINO GmbH has compiled this user information under the intention to support the user in the operation of the unit and to facilitate the application of the unit. For damages, which can be attributed to incorrect application or for alterations of the unit, the FINO GmbH is not responsible. A usage of the unit, which is not in accordance with the specifications, causes the loss of the guarantee. According to its policy the FINO GmbH permanently improves its products. FINO reserves the right to change or improve the products described in this documentation or change the documents itself without prior notice. FINO cannot be held liable for the completeness or correctness of the contents of this document.

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