 **FINOSCAN**
Spray 1

 **55554**

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 **55554**

ENG Instructions for use

 Please read carefully before use
Keep for referral purposes

 Only to be used by qualified
personnel!

1. Intended purpose
Titanium dioxide-free, micro-pulverizing scan
spray.

2. Indication
For the improvement of the optical charac-
teristics of prepared models when taken by
camera or scanner in CAD/CAM technology.
For intraoral and extraoral use.
Universally applicable for all CAD/CAM sys-
tems.

2.1 Contraindication
Do not use if incompatibilities or allergies to
constituents have been confirmed.

3. Fields of application
Only for processing in the dental laboratory
according to the intended use.

4. Safety precautions

 Danger

H222 Extremely flammable aerosol.
H229 Pressurised container: May burst if he-
ated.

H412 Harmful to aquatic life with long lasting
effects.

4.1 Prevention
P210 Keep away from heat, hot surfaces,
sparks, open flames and other ignition
sources. No smoking.
P211 Do not spray on an open flame or other
ignition source.
P251 Do not pierce or burn, even after use.
P260 Do not breathe Aerosols.
P271 Use only outdoors or in a well-venti-
lated area.
P273 Avoid release to the environment.
Ensure good ventilation and venting of the
work areas.

5. Processing instructions

 **SHAKE WELL** Shake well before use!

Place the spray cannula on the spray can.
Clean and dry the surface to be sprayed
thoroughly.
Spray the spray vertically from a distance of
5.0 cm - 8.0 cm onto the dry, clean surface.
Push the spray head through completely!
Do not use on large, heavily bleeding
wounds.
Vacuum off the aerosol with a saliva aspirator
during application. Do not inhale aerosol! It is
recommended to wear a mouthguard.
For intraoral use, briefly blow dry the pigment
film in the mouth before taking a CAD/CAM
image.
Do not start taking the picture until it has
dried. After taking the picture, simply clean
the surface with water or air.
Replace the cannula/nozzle after each intra-
oral application.

**The nebulizing nozzle of FINOSCAN
Spray 1 does not clog and therefore does
not have to be cleaned.**

6. Storage
P402+P404 Store in a dry place. Store in a
closed container.
P410+P412 Protect from sunlight. Do not ex-
pose to temperatures exceeding 50 °C.
Observe official regulations on storing pack-
agings with pressurized containers.

7. Physical data
Boiling range -11.7 °C; melting point:
-159.4 °C; ignition temperature 460 °C; va-
pour pressure 3000 hPa; density
0.64 g/cm³; max. VOC content >95 %; grain
size 1 to 50 µm

7.1 Composition:
8,5 % by mass of the contents are flammable.
titanium dioxide-free.

8. Delivery forms
FINOSCAN Spray 1
net 50 ml (gross 75 ml) 55554

9. Guarantee
Our technical recommendations of applica-
tion are based on our own experiences and
tests and should only be regarded as guide-
lines. It rests with the skills and experience of
the user to verify that the products supplied
by us are suitable for the intended proce-
dures. Our products are undergoing a contin-
uous further development. We reserve the
right of changes in construction and com-
position. It is understood that we guarantee the
impeccable quality of our products.

10. Disposal
P501 Dispose of contents/container in ac-
cordance with local/regional/national/interna-
tional regulations.

10.1 EWC waste code product
160504 WASTES NOT OTHERWISE SPE-
CIFIED IN THE LIST; gases in pressure con-
tainers and discarded chemicals; gases in
pressure containers (including halons) con-
taining hazardous substances; hazardous
waste.

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