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FINOCLEAN
Tartar and Cement Remover

69014

English

Instructions for use

Please read carefully before use
Keep for referral purposes

1. Fields of application
 Intensive cleaning of dentures, instruments, acrylic prostheses and impression trays made of stainless material. Removes contamination such as, for example, calculus, cement, impression materials and zinc oxide eugenol pastes. Only for extraoral use!

2. Safety precautions

 Danger
 H302 Harmful if swallowed.
 H314 Causes severe skin burns and eye damage.
 H335 May cause respiratory irritation.

2.1 Prevention
 P260 Do not breathe dust/fume/gas/mist/vapours/spray.
 P280 Wear protective gloves/eye protection.

2.2 Response
 P303+P361+P353 IF ON SKIN (or hair): Remove/Take off immediately all contaminated clothing. Rinse skin with water/shower.
 P305+P351+P338 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.
 P310 Immediately call a POISON CENTER or doctor/physician.
 P304+P340 IF INHALED: Remove victim to fresh air and keep at rest in a position comfortable for breathing.
 P301+P330+P331 IF SWALLOWED: rinse mouth. Do NOT induce vomiting.

2.3 Storage
 P403+P233 Store in a well-ventilated place. Keep container tightly closed.

2.4 Disposal
 P501 Dispose of contents/container in accordance with local/regional/national/international regulations.

3. Processing instructions

 Caution!
 Do not use on aluminium parts.
 Do not use vessels made of polycarbonate, stainless steel or acrylic glass.
 Always keep the solution covered and provide adequate ventilation.

 Removal of minor residues of temporary cements on instruments: soak cotton pad with FINOCLEAN and clean objects. Then rinse thoroughly with water.

Removal of larger residues of cement and calculus: fill FINOCLEAN into a suitable vessel and leave objects in vessel for at least 10 min. Cleaning in an ultrasonic bath will accelerate dissolving of contamination. Rinse objects thoroughly with water after cleaning.

 Removal of adhesive and impression residues from impression trays: fill FINOCLEAN into a suitable vessel and leave objects in vessel overnight. Rinse objects thoroughly with water after cleaning.

 Removal of stubborn residues of temporary cements, calculus or impression materials on thermally stable acrylic parts and prostheses.

 Caution!
 Place objects on a plaster model to avoid deformation!

Fill FINOCLEAN into a glass or porcelain vessel and place objects to be cleaned in vessel. Heat liquid to boiling temperature. Remove objects after 10 min. and rinse thoroughly with water.

4. Storage
 P233 Keep container tightly closed.
 P102 Keep out of reach of children.

5. Physical data
 Water-soluble and incombustible. Contains hydrochloric acid. 648/2004/EC: < 5 % non-ionic surfactants.

6. Delivery forms
 FINOCLEAN tartar and cement remover
 1000 ml 69014

7. Guarantee
 Our technical recommendations of application are based on our own experiences and tests and should only be regarded as guidelines. It rests with the skills and experience of the user to verify that the products supplied by us are suitable for the intended procedures. Our products are undergoing a continuous further development. We reserve the right of changes in construction and composition. It is understood that we guarantee the impeccable quality of our products.

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