FINOSEPT
Abformdesinfektion

88217

CE 0483

Deutsch

Gebrauchsinformation

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FINOSEPT
Impression Disinfection

88217

CE 0483

English

User Information

1. Fields of application
For disinfection and cleaning of all dental impression materials and dental technological workpieces made from silicone, polyether, alginate, hydrocolloid etc. Ready-to-use solution with quaternary ammonium salts. Free of aldehydes.

2. Safety precautions
No particular hazards worth mentioning.

3. Processing instructions
Rinse impression under running water and submerge subsequently in the disinfecting solution (impression must be fully covered by the liquid). Remove impression after two minutes and rinse with water.
Submerge impressions made from alginates and hydrocolloids only briefly in the disinfecting solution. Subsequently dry for a minimum of two minutes.
Also suitable for use in an ultrasonic bath. Do not mix with cleaning agents or other disinfecting agents.
Change impression disinfection daily. Useful life without load two weeks.

4. Storage
Store tightly sealed.

5. Physical data
visual appearance: clear solution
density: 0.994 g/ml
pH-value: approx. 7
aroma: lemon

6. Delivery forms
FINOSEPT impression disinfection
can 5000 ml 88217

7. Guarantee
Our technical recommendations of application are based on our own experiences and tests and should only be regarded as guidelines. It rests with the skills and experience of the user to verify that the products supplied by us are suitable for the intended procedures. Our products are undergoing a continuous further development. We reserve the right of changes in construction and composition. It is understood that we guarantee the impeccable quality of our products.

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