Short Instructions

PrograScan[®]
PS7

Laboratory scanner

Making People Smile

ivoclar

PrograScan® PS7

[en]	Short Instructions Laboratory Scanner	4
[de]	Kurzanleitung Laborscanner	6
[fr]	Mode d'emploi simplifié Scanner de laboratoire	8
[it]	Istruzioni d'uso brevi Scanner da laboratorio	10
[es]	Instrucciones resumidas Escáner de laboratorio	12
[pt]	Instruções de Uso resumidas Scanner de Laboratório	14


Step 1

Scope of delivery

Carefully unpack the product from the packaging and check the contents to make sure you have received all the parts included in the delivery form.

The scope of delivery includes:


- 1x PrograScan® PS7
- 1x Calibration body autodetection
- 1x Articulator base autodetection
- 1x Multi-die holder autodetection
- 2x Model mounting Blu Tack autodetection
- 1x Impression holder autodetection
- Incl. power cable and power supply unit
- Incl. 2x USB-C cable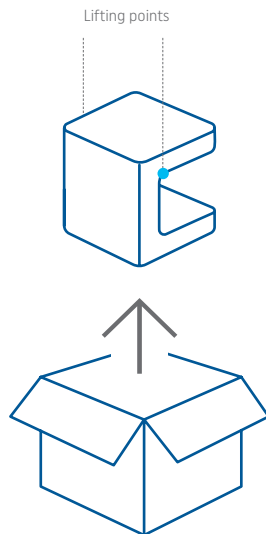

Please keep both the inner and outer packaging of the scanner for possible returns (repairs, trade-in, etc.).

Step 2

Connection

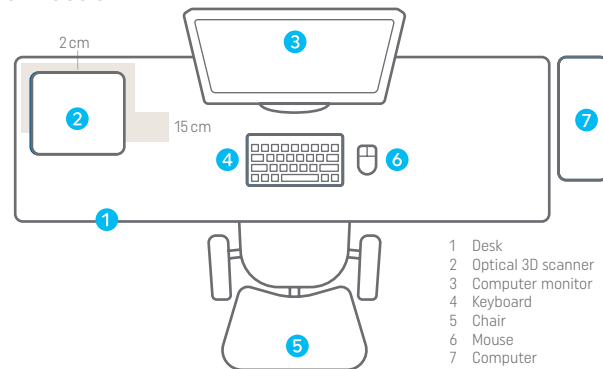Connections - scanner

- Power switch
- Power cable
- USB 3.0

Connections - PC

- USB 3.0 for scanner
- USB for keyboard/mouse
- Ethernet/LAN for internet

The figure shows a schematic representation of a complete system. Make sure to lay the cables along walls.

Step 3

Initial operation

After you have set up and connected the scanner correctly, you can start using it.

1. Switch on PC

Wait until the PC has completely booted up before you switch on the scanner.

2. Switch on scanner

Switch on the scanner and open the scan software.

3. Calibrate scanner

Calibrate the scanner in the scan software before first use.

We recommend you calibrate the scanner on a regular basis (1x per week).

If you get stuck

contact our support

If you have technical questions about the scanner, please contact our technical support. They will assist you with all their expertise.

You can also download our detailed Operating Instructions from our website. <https://www.ivoclar.com/> go to Instructions for Use.

Thank you for choosing our product.

We wish you much joy and success with it.

Only the systems supplied by us guarantee unrestricted functionality. Third-party systems may cause errors and are not recommended.

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