SHOFU

VINTAGE ZR

07.0:2020-09
72489

Manufacturer

SHOFU INC.

11 Kamitakamatsu-cho, Fukuine,
Higashiyama-ku, Kyoto 605-0983, Japan
www.shofu.com

SHOFU DENTAL CORPORATION

1225 Stone Drive, San Marcos,
California 92078-4059, USA

SHOFU DENTAL ASIA-PACIFIC PTE.LTD.

10 Science Park Road, #03-12,
The Alpha, Science Park II, Singapore 117684

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Firing in the air

PYSICAL PROPERTIES

Coefficient of thermal expansion and Glass transition point

	Coefficient of thermal expansion (25-500 °C)	Glass transition point
Opaque Liner	2.Firing 4.Firing	9.3×10 ⁻⁶ K ⁻¹ 620 °C
Margin	2.Firing 4.Firing	9.3×10 ⁻⁶ K ⁻¹ 635 °C
Body	2.Firing 4.Firing	9.4×10 ⁻⁶ K ⁻¹ 605 °C
Cervical Trans	2.Firing 4.Firing	9.4×10 ⁻⁶ K ⁻¹ 595 °C
Correction	2.Firing 4.Firing	9.4×10 ⁻⁶ K ⁻¹ 585 °C

Solubility Test (ISO Specification : Under 100 µg/cm²)

	Solubility amount (µg/cm ²)
Body, Enamel	14.8

en

ZR Bond Porcelain

READ CAREFULLY
BEFORE USE

INDICATIONS

High fusing porcelain for fabrication of Zirconia ceramic restorations

PRECAUTIONS

1. If any inflammation or other allergic reactions occur on operator, immediately discontinue use and seek medical advice.

2. Avoid contact with eyes. In case of contact with eyes, immediately flush the eyes with plenty of water and seek medical advice.

3. Wear protective glasses etc while grinding and polishing this product to avoid damages to eyes.

4. Use local dust extractor, dust protective mask, etc while grinding this product to avoid harmful influence of the dust on the human body.

5. This product is intended for use by dental professionals only.

DIRECTIONS FOR USE

Refer to the separate instruction manual and step card.

NOTE ON USE

1. Optimum firing schedule is different depending on the type and model of the furnace. Try to determine the optimal conditions at which every restoration is fired completely.

2. When mixing CLM (Clear Margin) 40 % or more with basic margin porcelain, firing temperature should be lowered. When CLM is used solely, firing temperature should be 980 °C/1,796 °F.

3. The Body build-up should be done to the final anatomical shape of the tooth in order to ensure the exact dimension for the Enamel porcelain.

4. Drying time should be altered depending on the size of the restoration to be fired. (Restorations might explode inside the furnace if moisture remains inside.)

5. Enamel porcelain should be built up oversized in consideration of firing shrinkage.

6. When building up additional porcelain onto the dried area, wet the area with water moistened brush in advance.

7. Mix Opaque Liner before use to avoid separation between powder and liquid.

8. Follow the instructions for use of each dental material or instrument to be used in conjunction with this product

9. Do not use this product for any purposes other than specifically outlined in INDICATIONS in these instructions for use.

STORAGE

Store at room temperature(1-30 °C/34-86 °F). Keep away from direct sunlight.

CAUTION: US Federal law restricts this device to sale by or on the order of a dental professional.

Location of the symbols glossary: https://www.shofu.com/en/symbols_glossary/

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